

# Boiler O-ring Installation Sheet [002-1080-00]

<b>Components:</b>	
Boiler .....	1
O-ring .....	1
<b>Special Tools:</b>	
none	

**Caution:** Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool then drain reservoir.

**Step 4:** Remove boiler top.


**Step 2:** Remove Pressure Relief Valve Enclosure.

**Step 5:** Remove gasket.

**Equipment Alert**  
Pull ring on relief valve to release any pressure in chamber.

**Step 1:** Remove pressure relief hose.

**Note:** Inspect and clean mounting surface and place new gasket and secure.

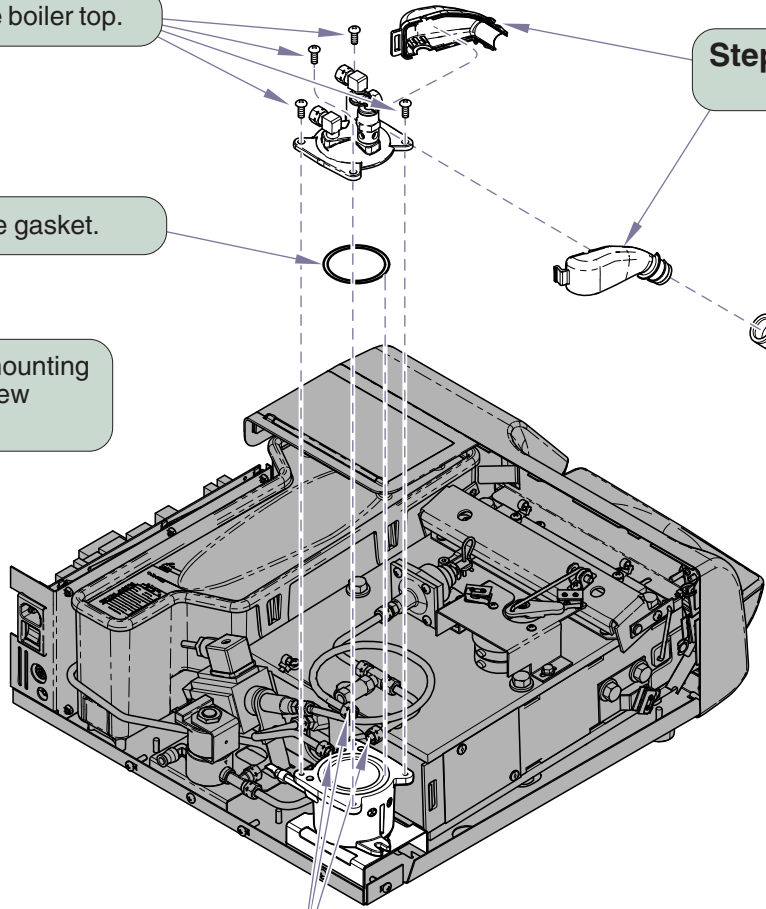

**Step 3:** Disconnect copper tubes, and neoprene tube from boiler.

**Installation:** Reverse steps for installation.