

# Thermostat Installation Sheet [002-1089-00]

**Components:**  
Boiler assembly .. 1  
Thermostat ..... 1

**Special Tools:**  
none

**Caution:**  
Always disconnect power to sterilizer before removing cover. Allow sterilizer to cool.


**Equipment Alert**  
Pull ring on relief valve to release any pressure in chamber.

**Step 1:** Check reset button on boiler thermostat. Push in to reset.

**Step 3 :** Remove 2 screws from boiler bracket.

**Step 4:** Lift up boiler and unhook wiring from thermostat and remove.

**Step 2:** Disconnect copper tubing, neoprene tube, and wires from boiler.

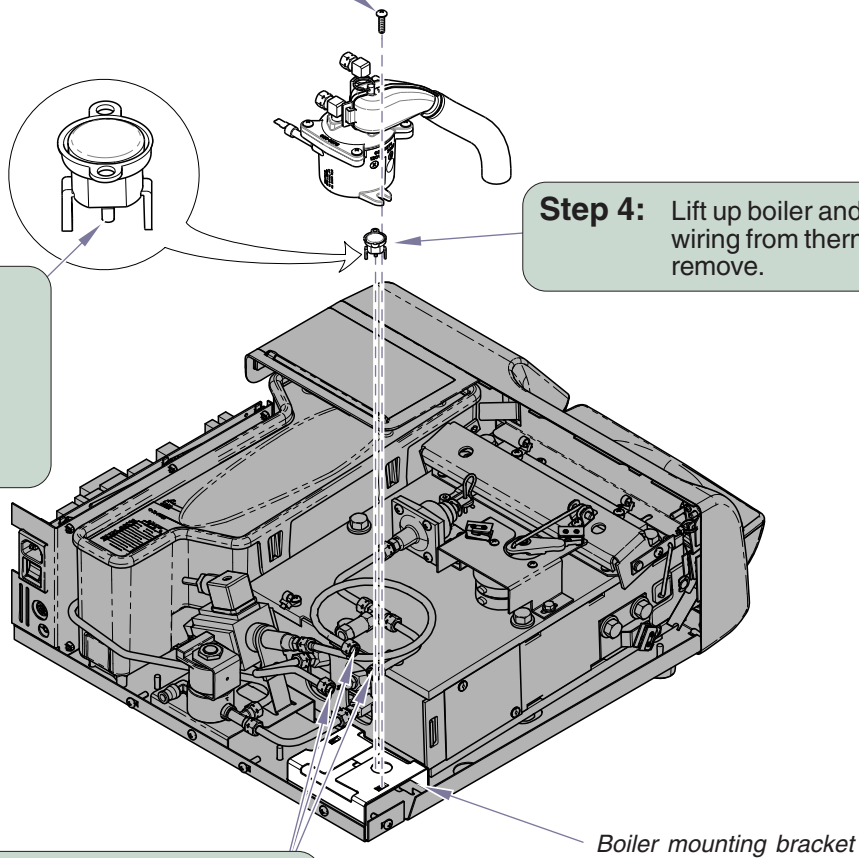

**Installation:** Reverse steps for installation.