

Dressing Nook Door Reversal Instructions

Applies to Models:
Dressing Nooks

Special Tools:
none

Note
This procedure reverses the “swing” of the door.

Step 2: Remove Flap Stay.
A) Remove screws (5) from flap stay.
B) Mount flap stay to opposite side of cabinet.

Step 1: Rotate the Mirror.
A) Remove screws (2) and washers (2) from mirror door handle.
B) Rotate mirror 180° and secure with washers and screws .

Step 3: Rotate the Door.
A) Remove screws (4) from door hinge.
B) Rotate Door 180°.
C) Secure hinges to other side of cabinet.
D) Mount flap stay to door.

