



To extend / retract the foot section...
 A) Squeeze handle on either side of foot section.
 B) Move foot section in / out to desired position.
 C) Release handle(s) to lock position.

Release Handles

To access the debris tray...
 Pull tray outward.

MA7905p



Brake Pedal

Rotational Base (optional)
To rotate the chair base...
 A. Press & release brake pedal.
 B. Rotate the chair to desired position.

To lock base in position...
 Press & release brake pedal.

Note: The base will lock automatically after two minutes.

MA7906p

Important Information

Max. Patient Weight: 450 lbs (204 kg)

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:

1-800-Midmark (1-800-643-6275)
 8:00am until 5:00 pm Monday through Friday (EST)

Midmark Corporation
 60 Vista Drive
 Versailles, OH 45380-0286
 Phone: 937-526-3662
 Fax: 937-526-5542
www.midmark.com



Because we care.



Quick Reference Guide
646 / 647 Barrier-Free®
Podiatry Procedures Chair

Model Number:

Serial Number:

Dealer Information:

Purchase Date:

Upholstery Color:

003-1893-00 Rev D 7/10/15

This guide outlines basic operation of the chair. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



MA7870p

To operate the Base / Back / Tilt functions...
Press & hold the appropriate button.

Example: The Base Up button is called out in the illustration.

Programming Chair Positions (647 programmable models only)
This feature allows you to easily recall frequently used chair positions.

To program a chair position...

- Move chair to desired position.
- Press the Program button. [single "beep"]
- Press the desired Position button (1, 2, 3, or 4). [three "beeps"]
(Tip: You must press the Position button within 5 seconds)



To recall a programmed position...

Press & hold the desired Position button (1, 2, 3, or 4).



Attention

To Stop movement of any automatic chair function:
Press the Stop button on the touch pad or foot control.

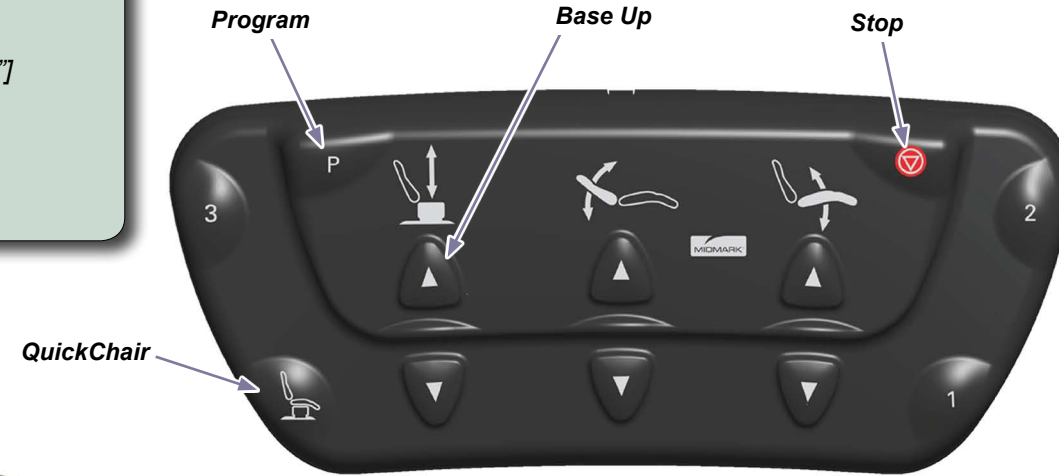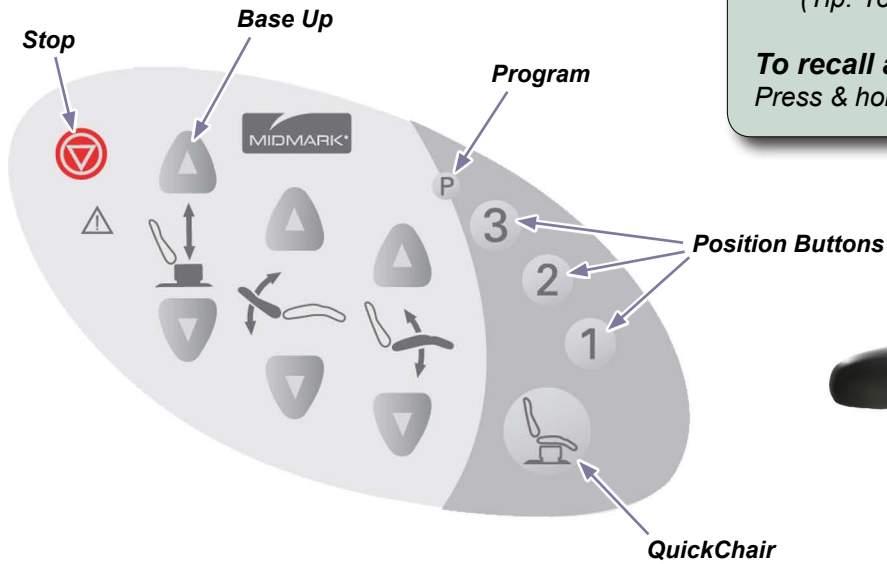

QuickChair Function

This function activates the Back Down, Tilt Down, and, Base Down (647 only) functions simultaneously.

To activate the QuickChair function...

Press & hold the QuickChair button.

Note: The 3-function, programmable controls are shown. The applicable symbols used on the 2-function, and non-programmable controls are the same.

QuickExam Function (647 non-programmable models only)

This function positions the chair for an exam by activating the Base Up and Tilt Up functions simultaneously.

To activate the QuickExam function...

- Press & hold the QuickExam button.
- Release button when chair reaches desired position.

