





# Sink Apron Panels Kit [002-1152-0x-xxx]

**Contents:**  
Installation ..... 1  
Exploded View ..... 2

**Special Tools:**  
Nutsert Insert Tool

**Intended Use:**  
To hide bottom of sink from public.

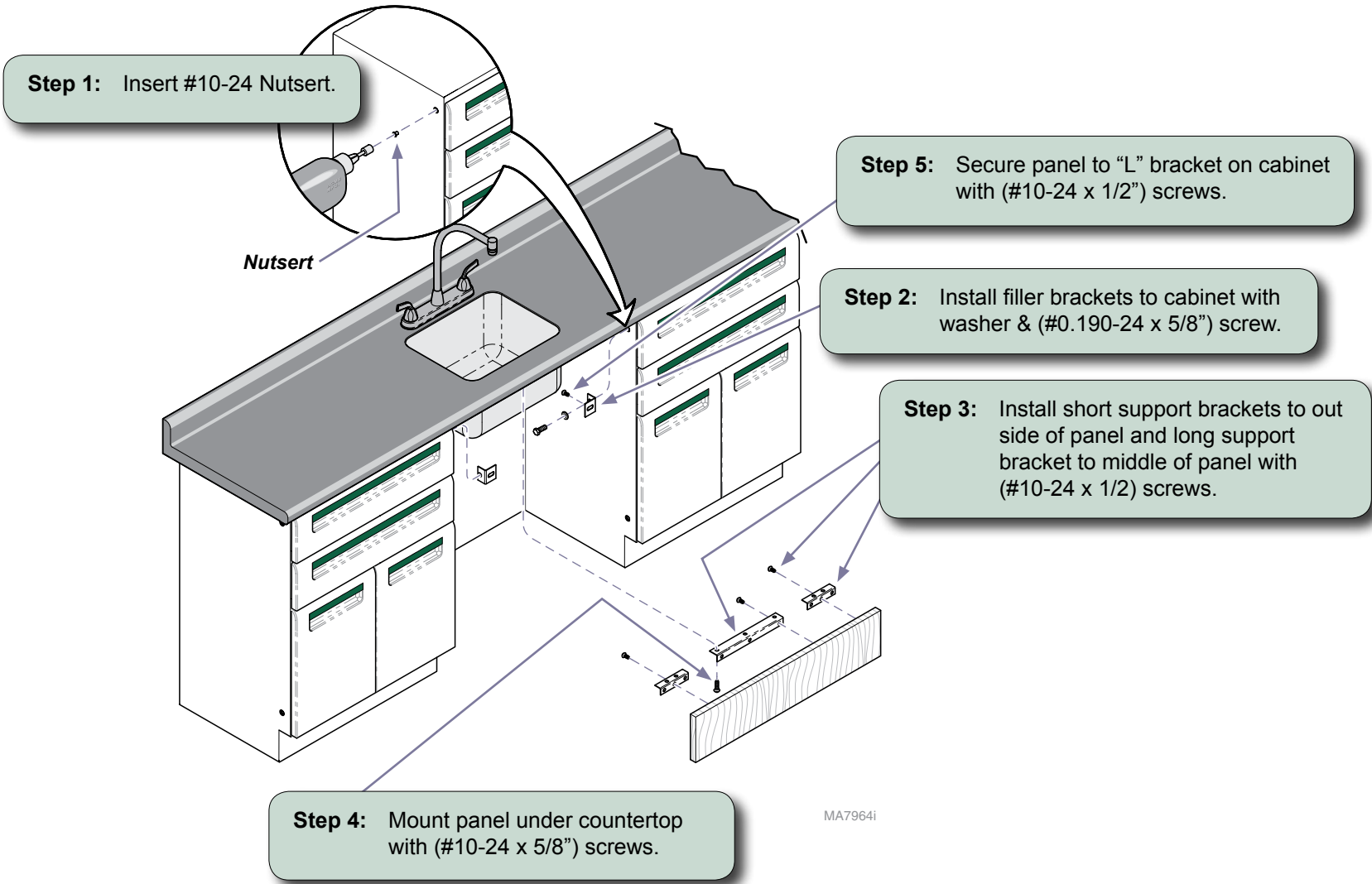

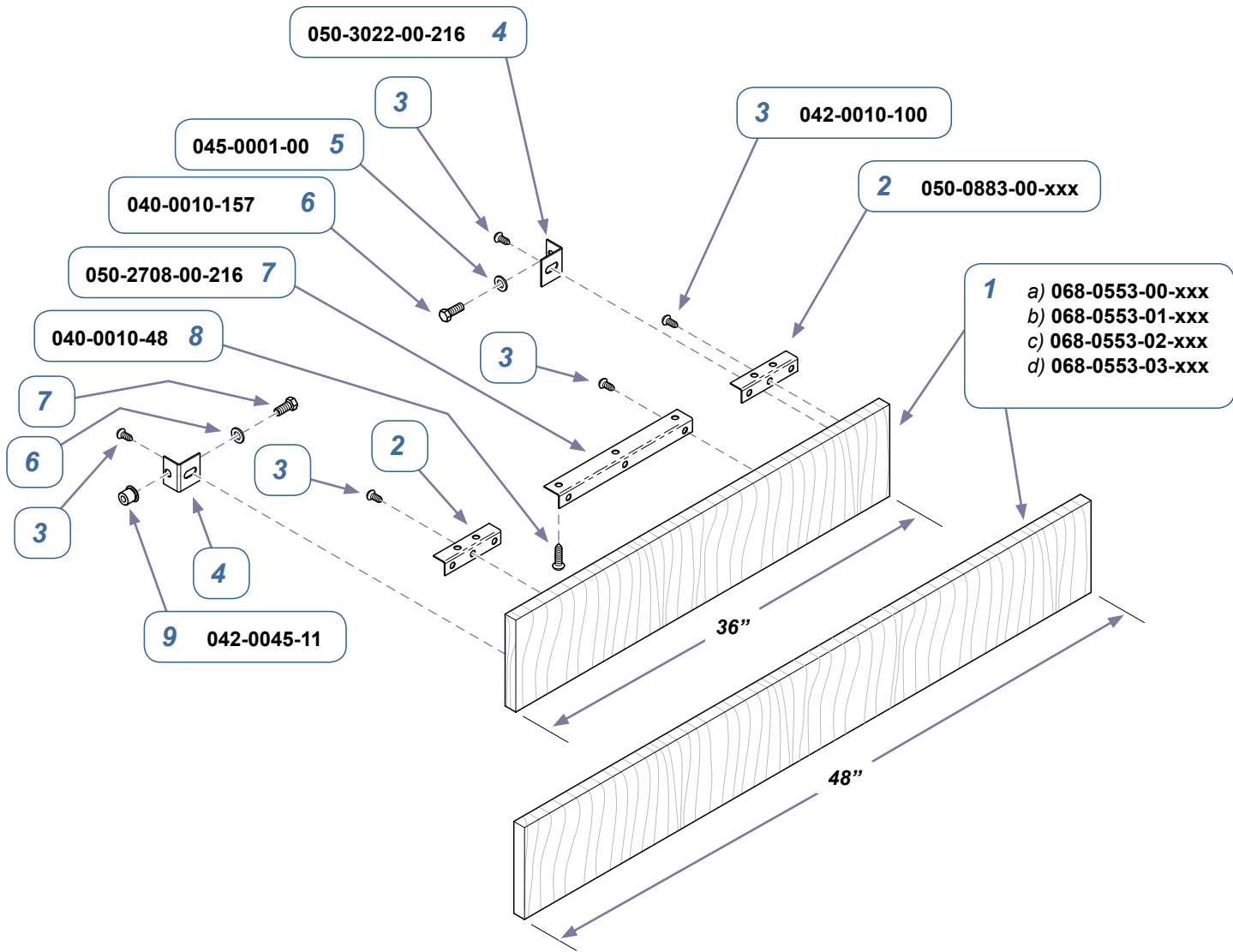

MA7965i

<b>002-1152-0x-xxx Sink Apron Panels Kit</b> (includes items 1 thru 10)		
Item	Description	Qty.
1	<ul style="list-style-type: none"> <li>• a) 36" Apron Panel (shown)</li> <li>• b) 48" Apron Panel (shown)</li> <li>• c) 42" Apron Panel</li> <li>• d) 30" Apron Panel</li> </ul>	1
2	• "L" Bracket	2
3	• Screw (#10-24 x 1/2")	9
4	• Top Filler Bracket	2
5	• Washer	2
6	• Screw (#0.190-24 x 5/8")	9
7	• Support Bracket	1
8	• Screw (#10-24 x 5/8)	2
9	• Nutsert	1

**Serial Numbers / Dates:** all