



# Pressure Relief Valve [002-1197-00]

**Intended Use:**  
To provide replacement for the pressure relief valve.



**Requirements / Restrictions:**  
1. Power off, unplug unit, allow to cool to ambient / room temperature and remove cover.  
2. Release pressure by pulling ring of pressure relief valve if necessary.

- Components:**
- Pressure relief valve w/ washer ..... 1
  - Enclosure (RH) for Pressure relief valve ..... 1
  - Enclosure (LH) for Pressure relief valve ..... 1
  - Reducer 1/2 NPT to 1/8 NPT ..... 1
- Special Tools:**  
none

**NOTE:** The Pressure Relief Valve and Reducer have a seal on the threads to prevent leakage.

**Step 2:** Remove Pressure Relief Valve Enclosures.

**Step 3:** Remove Pressure Relief Valve.

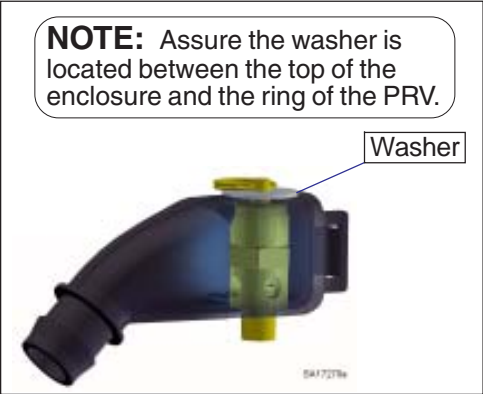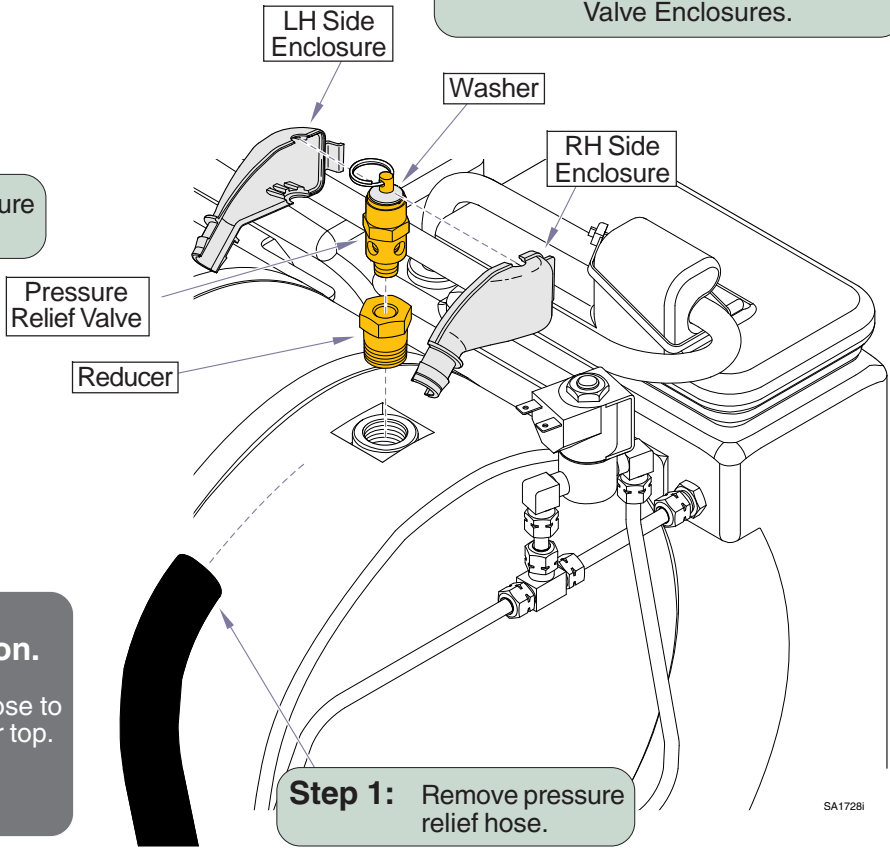

**Installation**  
**Reverse steps for Installation.**

Note: If necessary, trim the PRV Hose to prevent it from touching the counter top.

M9....13 3/4" (35 cm)  
M11..16 1/4" (41.25 cm)