

Elevance Upholstery Installation Guide

Applies to Models:
153810 (all)

Special Tools: none

Language of origin: English

WARNING
Always disconnect power before removing covers.
Failure to do so may result in personal injury.


Step 1: Insert seat tabs into mounting slots on chair frame and push downward on foot end. Rotate locking knob on seat bottom to secure.

Step 2: Insert headrest into mounting slot.

Step 3: Adjust headrest tension if necessary.
Loosen two hex screws and turn the friction cam adjustment bolt to exert the desired amount of tension on the headrest.

Tighten two hex screws to set the mechanism.

Equipment Alert
Use the friction cam adjustment mechanism to set the amount of tension necessary to raise/lower the headrest. Once this adjustment is set, the mechanism maintains that amount of tension.

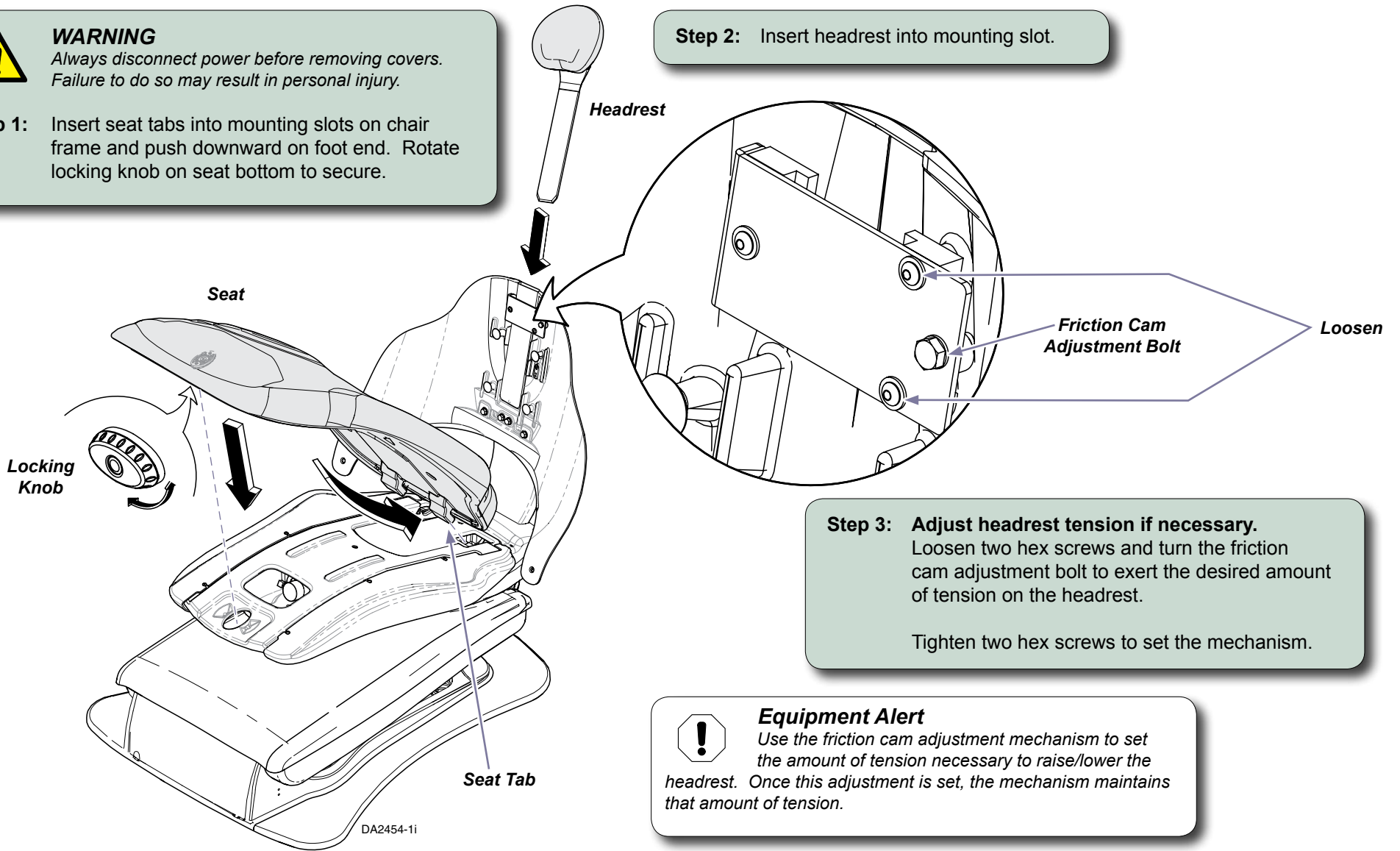


Step 4: Chairs without SerenEscape heat and massage option can go to Step 7.
Cut wire ties securing harnesses and gently pull out of storage cavities to make connections.

Step 5: Connect harnesses in chair back to back upholstery.

Step 6: Wire tie harnesses together and gently push any excess into back upholstery storage pocket.

Step 7: Align holes in back upholstery with back mounting posts and push downward to lock in place. Insert two screws thru chair arms and tighten to secure.

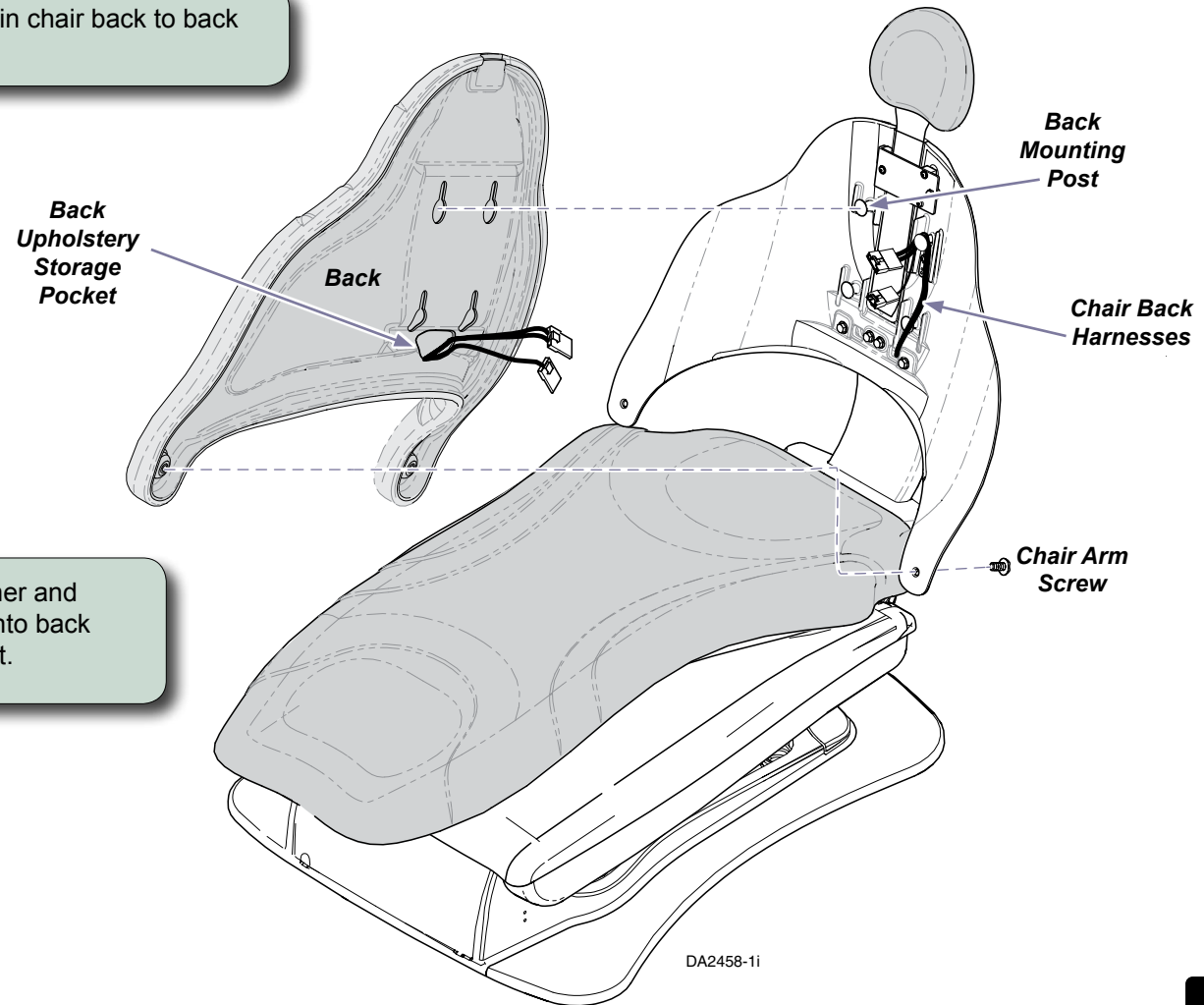

DA2458-1i

