



Welch-Allyn Light Bracket [9A464]


Contents:
Installation 1
Exploded View 2

Language of origin: English


Intended Use:
To mount a Welch-Allyn 44410 or 48810 light to models: 230 / 630 / 646 / 647

Restrictions:
Requires Base Rail or Accessory Rail (see accy. # below):
9A205 [Models 230, 630 (-001 thru -009), 646, 647]
9A557 [Model 630 (-010 thru -013), 630 (-020 thru -023)]

Step 1: Install hanger assembly
A) Loosen knob.
B) Slide hanger assembly onto base rail.
C) Tighten knob.

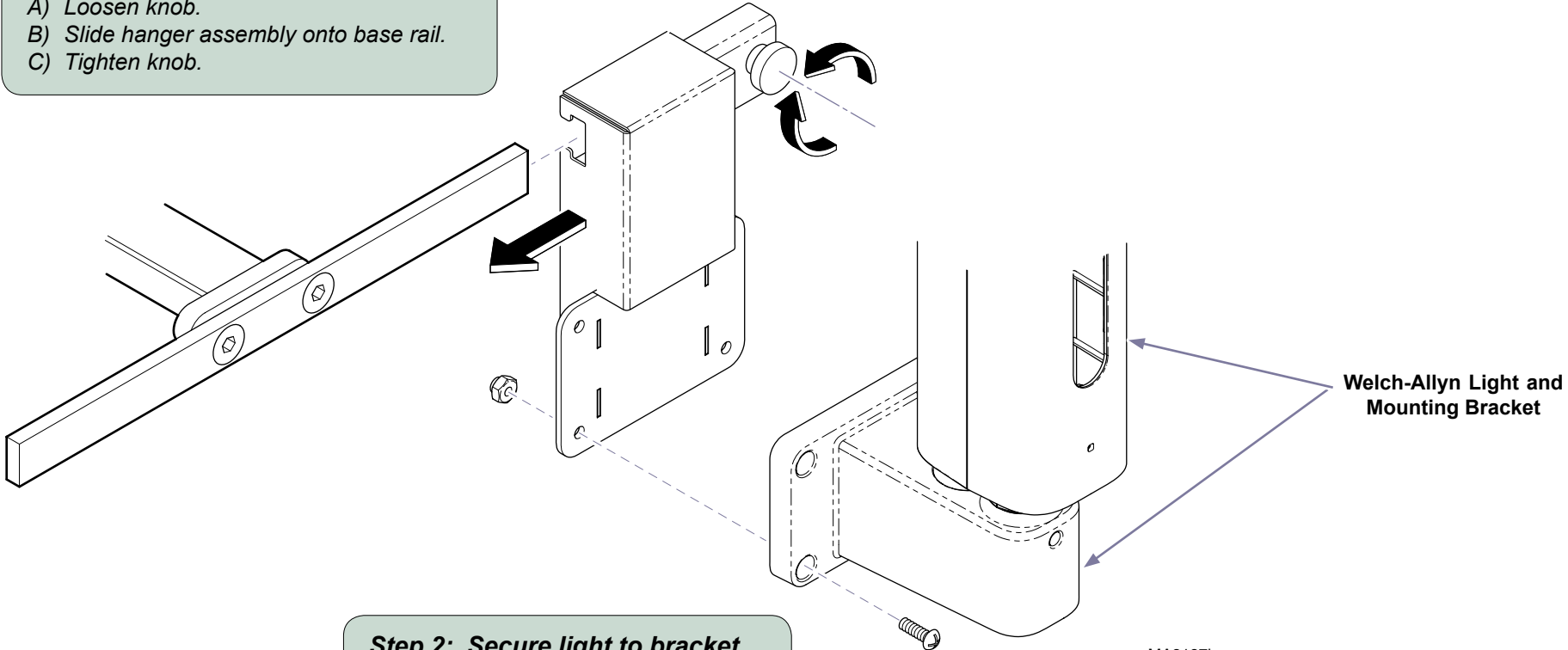

Step 2: Secure light to bracket.
A) Position light as shown.
B) Secure light with four screws / nuts.

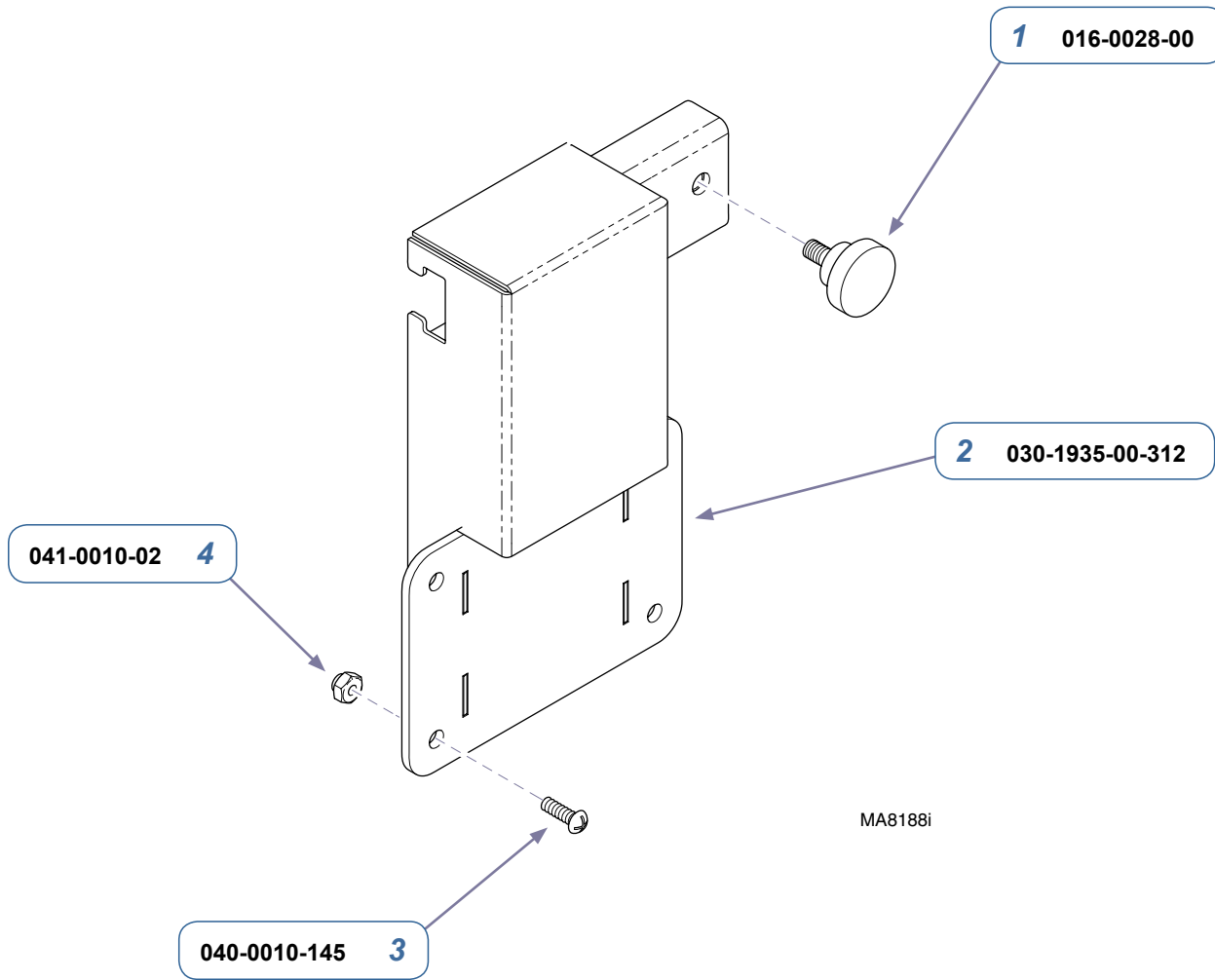

MA8188i

9A464001 Welch-Allyn Light Bracket (includes items 1 thru 4)		
Item	Description	Qty.
1	Knob	1
2	Light Bracket	1
3	Screw (#10-24 x 1/2")	4
4	Nylock Nut (#10-24)	4

Serial Numbers / Dates: all