



# Welch-Allyn Light Bracket [9A464]


**Contents:**  
Installation ..... 1  
Exploded View ..... 2

Language of origin: English

**Intended Use:**  
To mount a Welch-Allyn 44410 or 48810 light to models: 230 / 630 / 646 / 647


**Restrictions:**  
Requires Base Rail or Accessory Rail (see accy. # below):  
**9A205** [Models 230, 630 (-001 thru -009), 646, 647]  
**9A557** [Model 630 (-010 thru -013), 630 (-020 thru -023)]

**Step 1: Install hanger assembly**  
A) Loosen knob.  
B) Slide hanger assembly onto exam chair rail.  
C) Tighten knob.

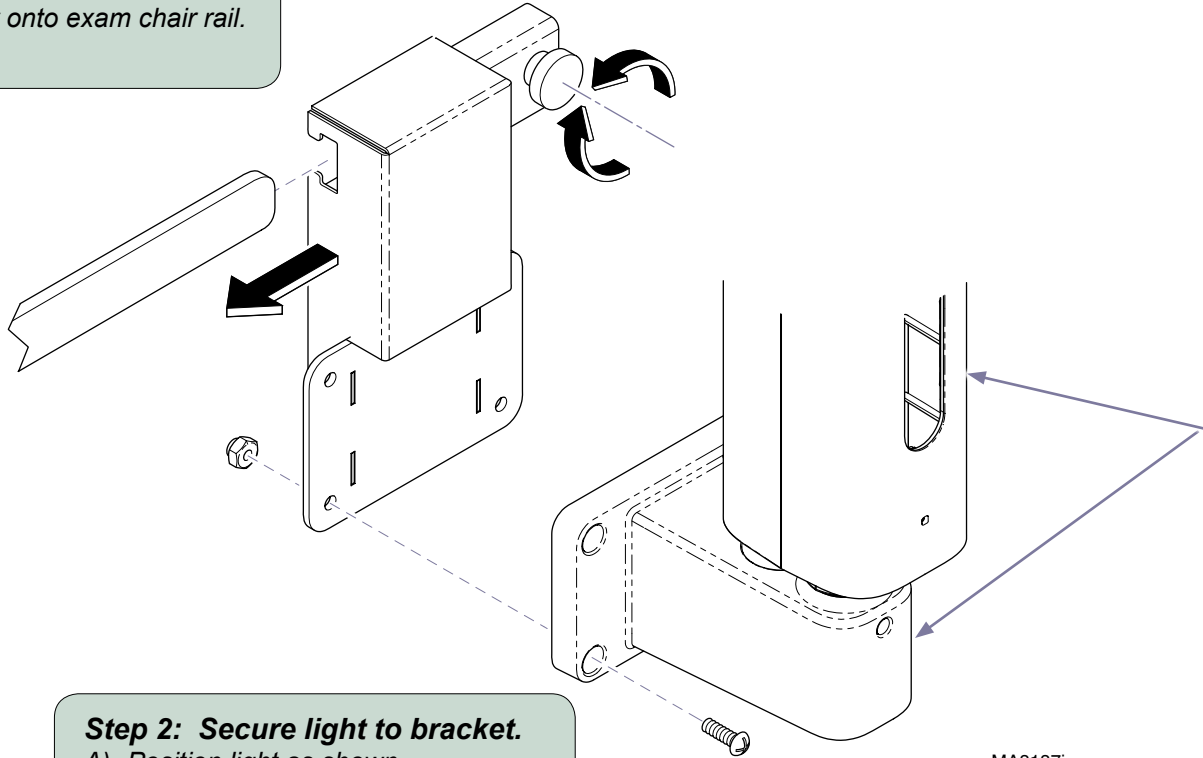

**Step 2: Secure light to bracket.**  
A) Position light as shown.  
B) Secure light with four screws / nuts.

MA81871

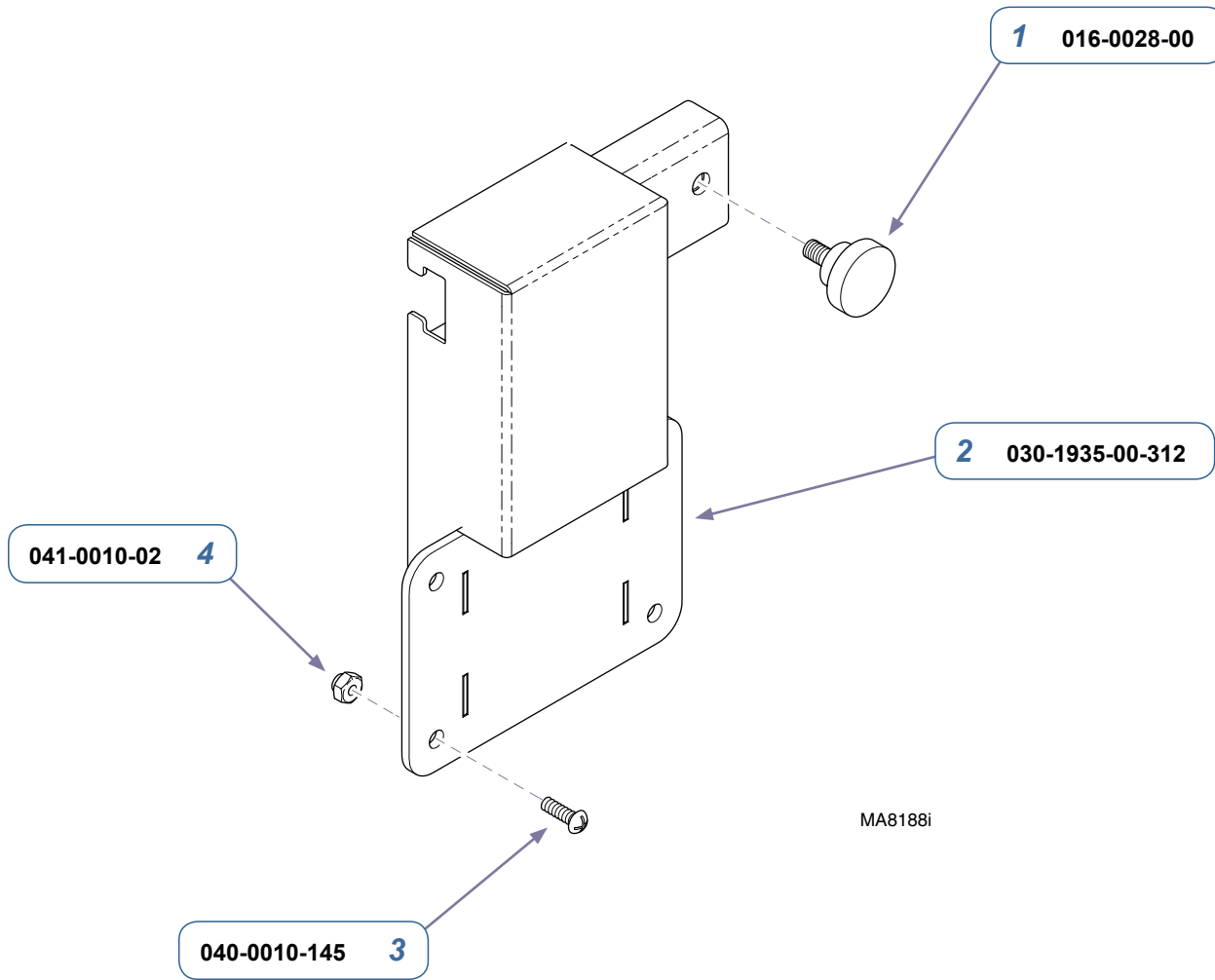

<b>9A464001 Welch-Allyn Light Bracket</b> (includes items 1 thru 4)		
Item	Description	Qty.
1	Knob	1
2	Light Bracket	1
3	Screw (#10-24 x 1/2")	4
4	Nylock Nut (#10-24)	4

**Serial Numbers / Dates:** all