


Cylinder Replacement

[002-1364-00 / 002-1365-00 & 003-10333-00]

Components:
Cylinder 1
Special Tools:
Rubber Mallet


- Step 1: Remove cylinder from stool top/base.**
- A) Gently tap metal weldment under seat to loosen stool top.
 - B) Remove stool top from cylinder.
 - C) Remove cylinder cover.
 - D) Hold cylinder while loosening using mallet.
 - E) Tap base to remove cylinder while holding cylinder.
 - F) Remove cylinder.

Equipment Alert
Do not strike plastic cover under seat when removing components. Failure to comply will cause damage.

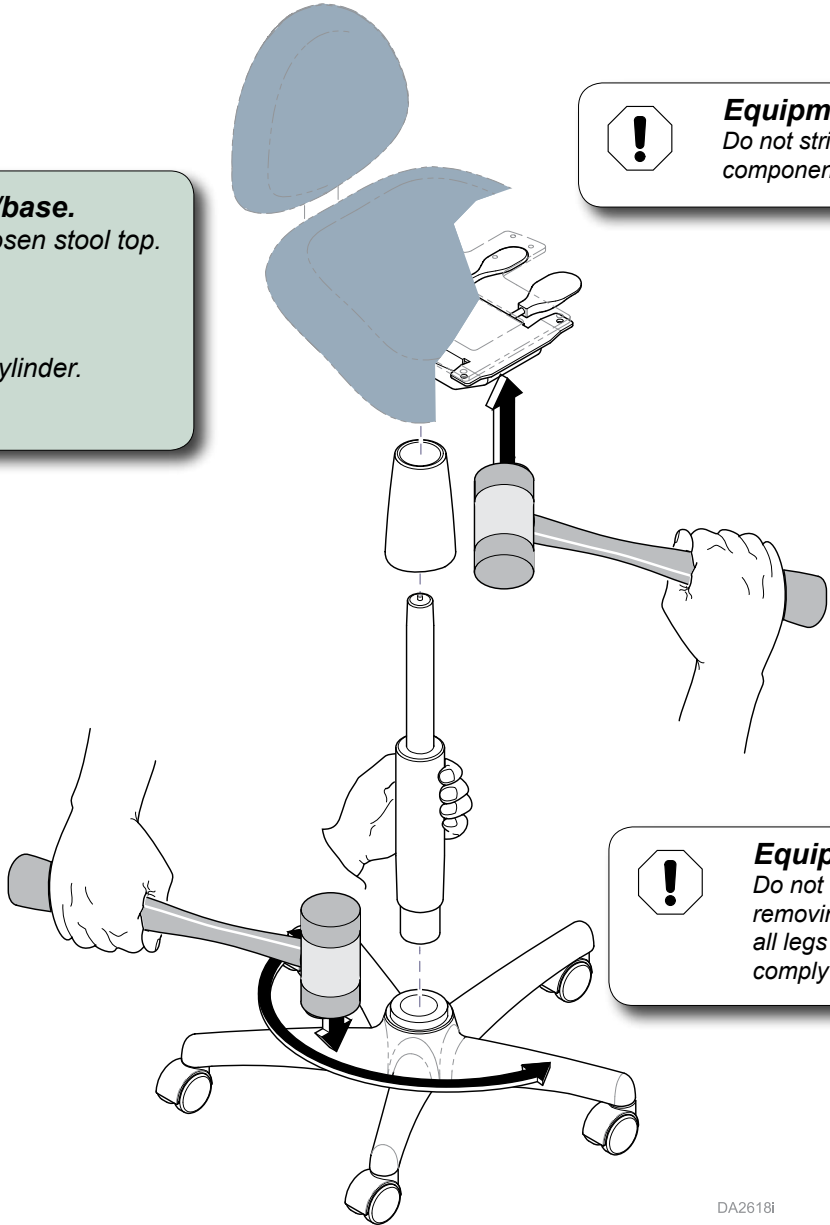

Equipment Alert
Do not strike only one leg on base when removing cylinder (see Step 1-E). Rotate strikes to all legs until base is removed from cylinder. Failure to comply will cause damage.

DA2618i

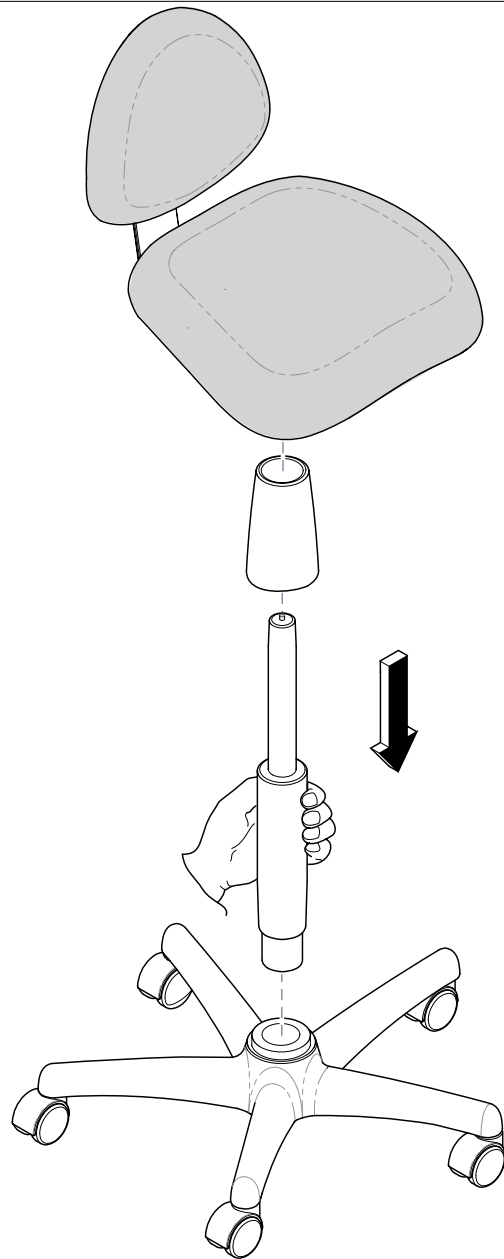

Step 2: Install new cylinder to base/stool top.

- A) Install new cylinder to base.
- B) Install cylinder cover.
- C) Press stool top on new cylinder and base.
- D) Sit on stool to secure cylinder.

DA2620i



Because we care.