


Monitor Arm Installation: *Post Clamp Mounted*

Applies to:
002-1836-00
Special Tools:
none

- Step 1: Install monitor arm bracket.**
- A) Disconnect power source from chair.
 - B) Position monitor arm bracket directly above access hole of the desired 2" diameter post.
 - C) Position the bracket clamp and secure with screws.
 - D) Install bracket cover.

Step 2: Insert monitor support arm into bracket.

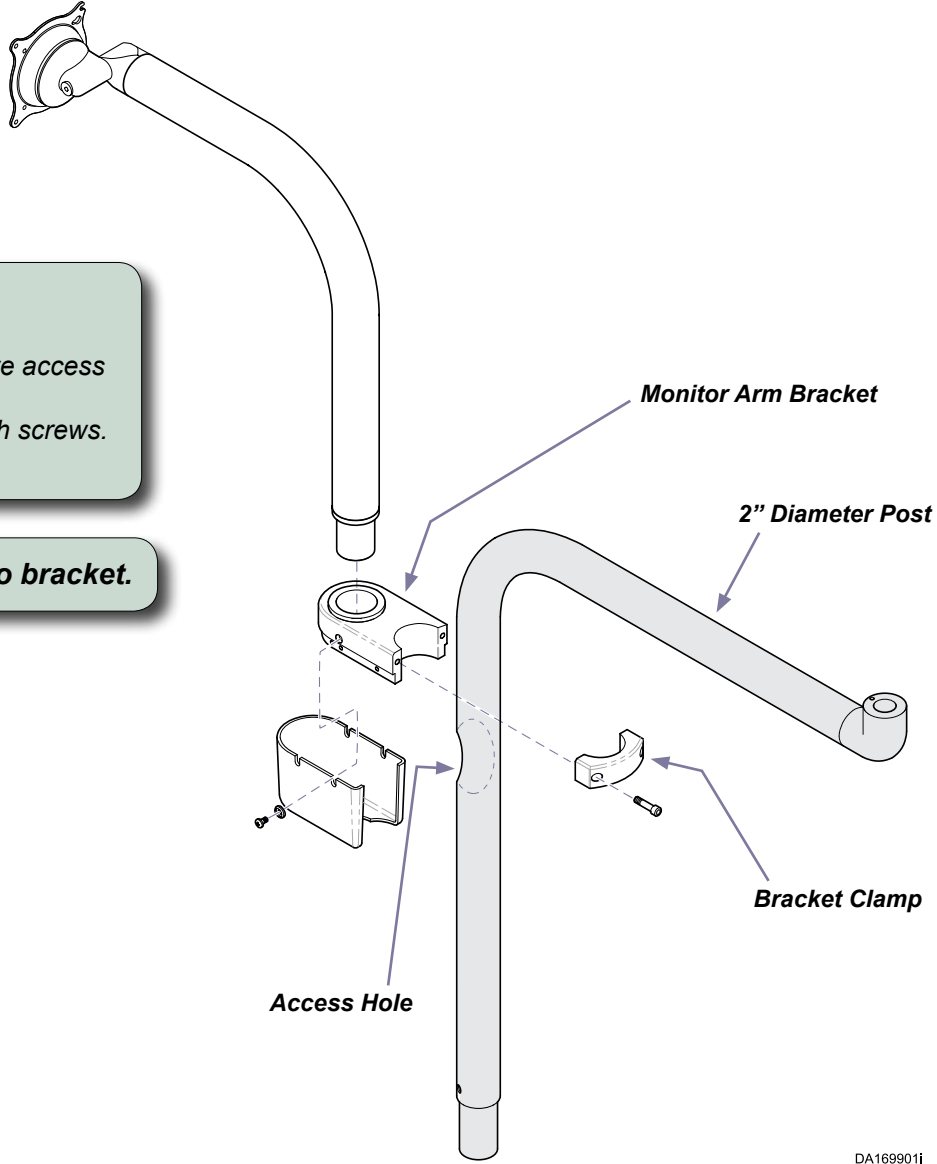

DA169901i



Equipment Alert

Maximum weight capacity of mounting plate is 15 lbs. (7 kg).

Step 3: Secure the monitor to mounting plate.


Note: Use the mounting screws supplied with the monitor.

Step 4: Route monitor cord.

- A) Route cord through access hole in upper arm.
- B) Pull cord through access hole in lower arm.
- C) Connect monitor cord per monitor manufacturer's instructions.
- D) Connect power source to chair.

If monitor will not stay in position...

- A) Loosen locking nut.
- B) Tighten tension screw as necessary.
- C) Tighten locking nut.

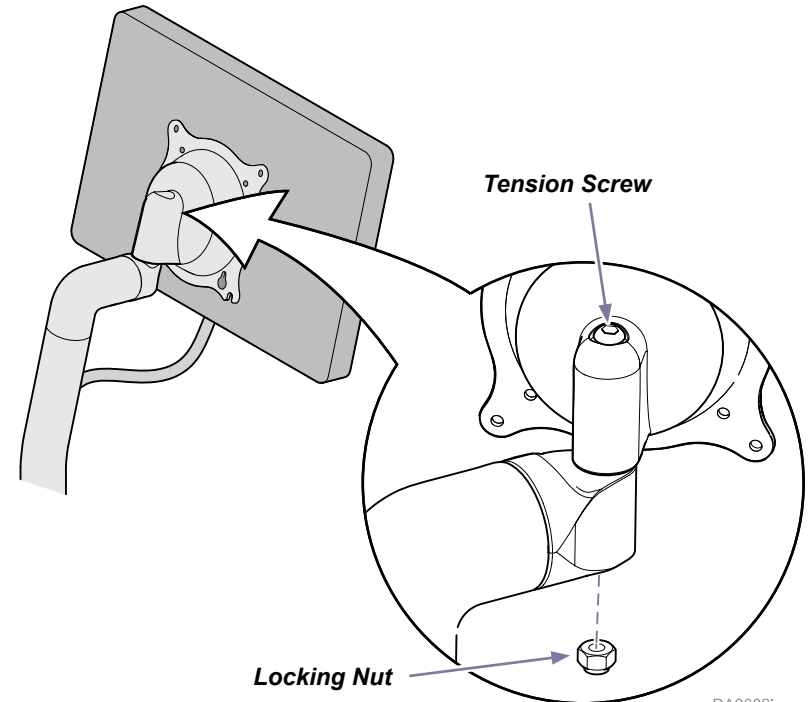

DA2682i