

PC Board Replacement Kit [002-1402-00]

Components:


PC Board.....	1
RTV Silicone.....	1

Step 1: Disconnect power to light, then remove housing cover.

Step 2: Disconnect wires.

Notches must be positioned at the top


Step 3: Remove two clamps.

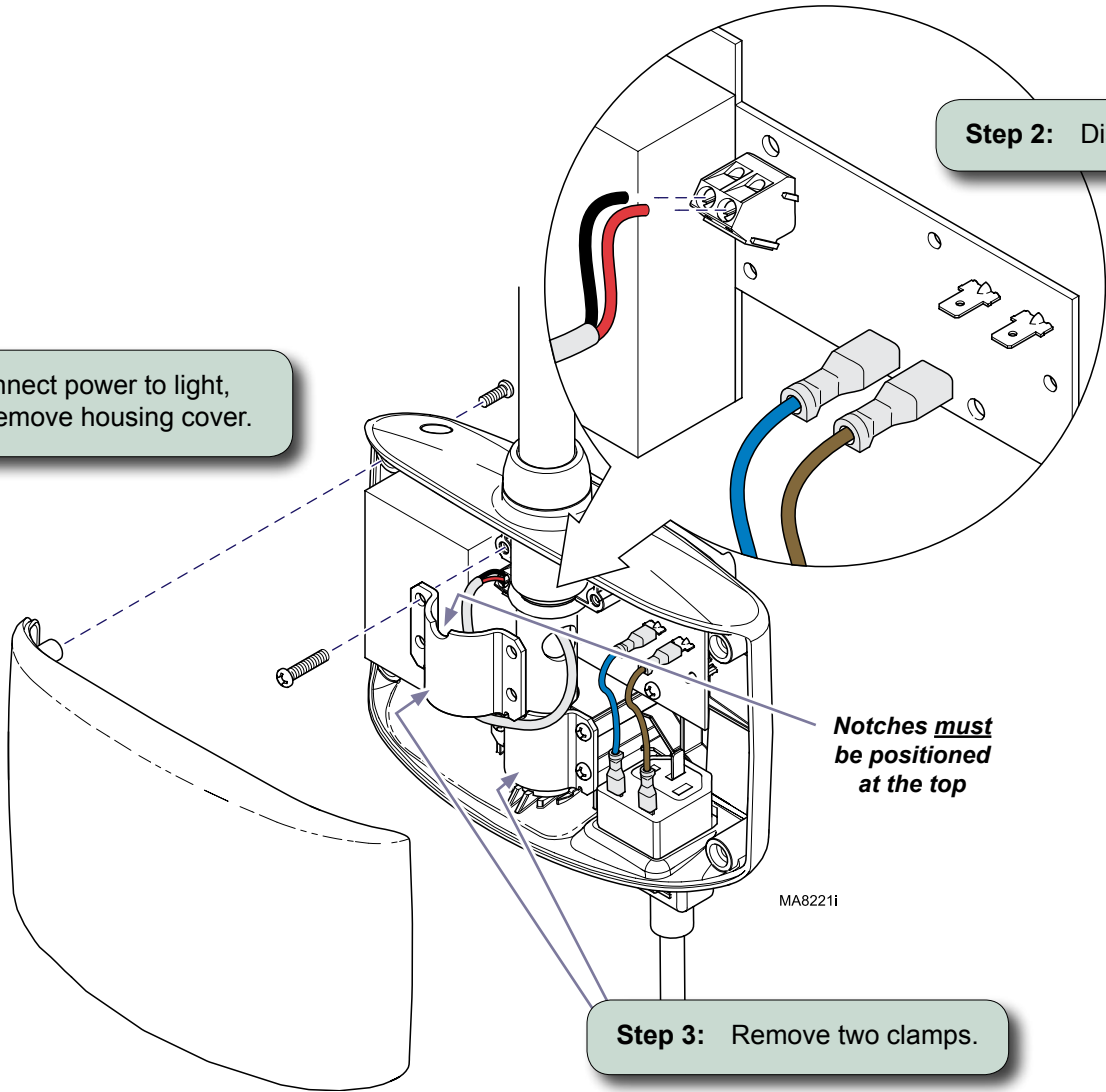

Step 4: Remove PC board from housing.

*Note: PC board is held in place with screws & silicone.
Pull / pry PC board to remove*

**Be sure light pipe stays in place
when removing PC board.**

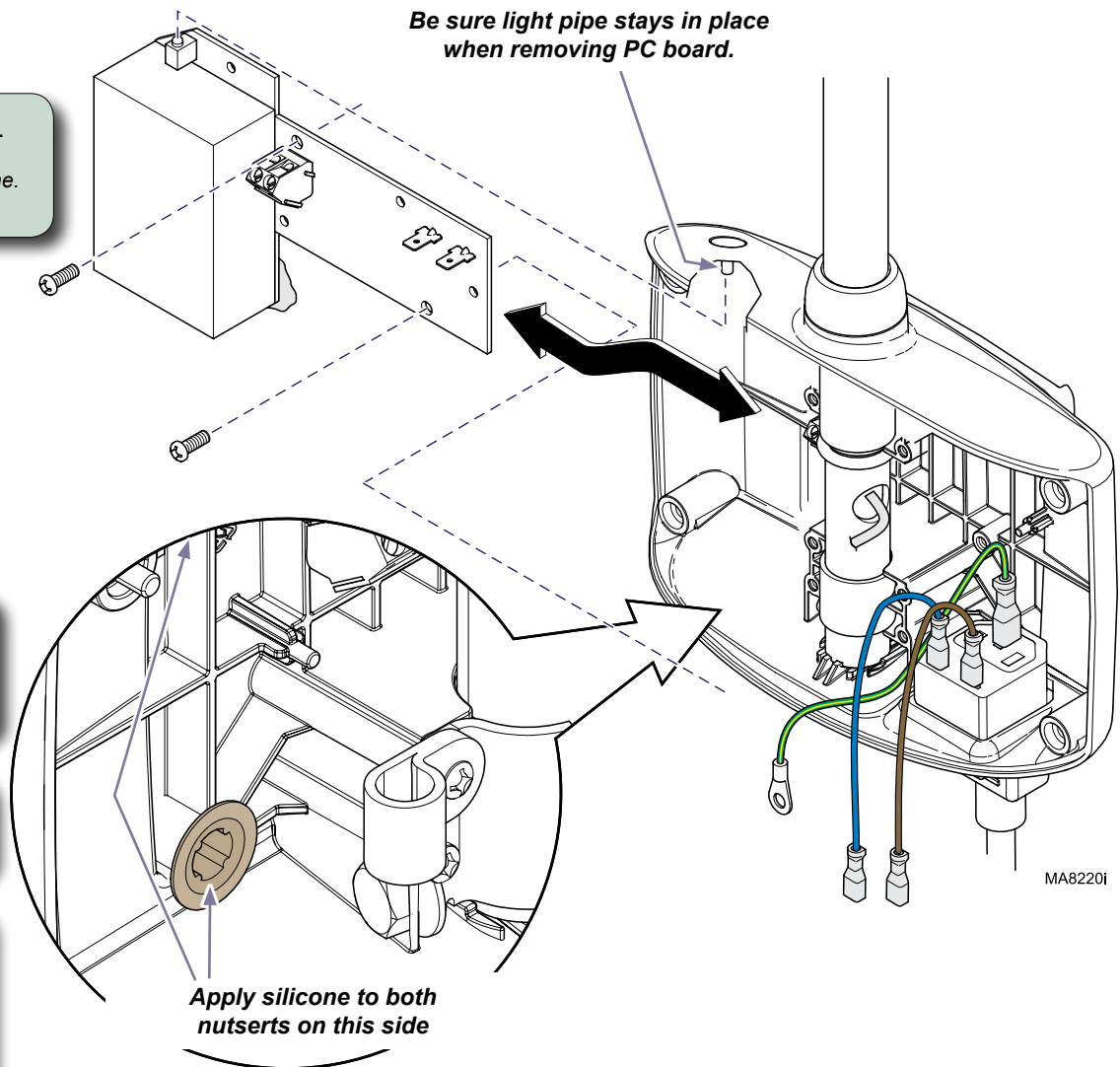

Step 5: Apply silicone to nutserts.

*Note: Only apply silicone to the two nutserts that are shown.
Do not allow compound to harden before assembling circuit board.*

Step 6: Position PC board,
Secure with two screws.

Step 7: Connect wiring.
Install two clamps.
Install housing cover.

Note: Refer to illustration on Page 1.