

SerenEscape Connector Kit [002-1430-00]


| |
|-----------------------|
| Components: |
| Retainer Clip..... 3 |
| Special Tools: |
| none |

Step 1: Remove upholstery.
A) Disconnect chair from power source.
B) Remove two screws in arms and lift up back to free from mounting posts.

WARNING
Always disconnect power to the chair before removing covers.

Step 2: Access connectors.
A) Access connectors from the chair back casting and upholstery.
B) Allow space for clip installation.

Mounting Post

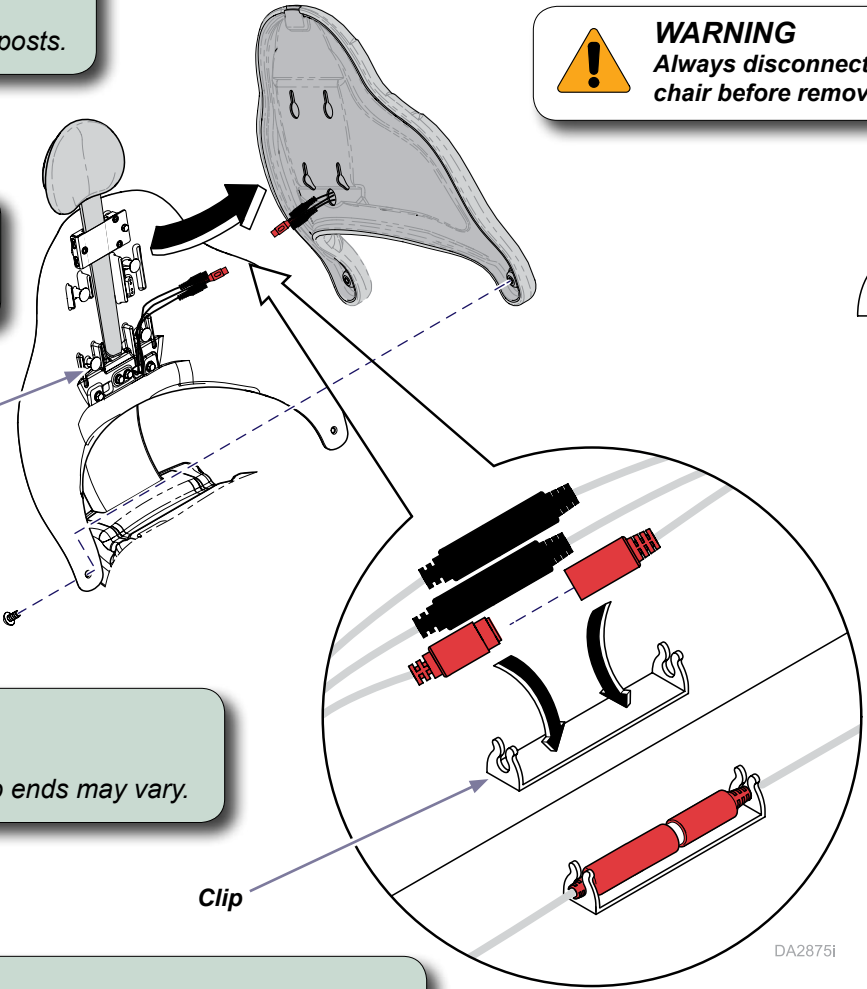

Step 3: Install clips.
A) Check connections (black to black and red to red).
B) Install one clip on each connection. Connection position at clip ends may vary.

Step 4: Install upholstery, connect power and test.
A) Place connections into chair back casting.
B) Keep connection wiring clear from obstructions during upholstery installation - reverse Step 1.
C) Test the heat and massage operations using the remote.

DA2875i