




Goose Neck Repair Kit [002-1529-00, 002-10211-00]

- Components:**
Flex Arm Assy..... 1
Parts Bag..... 1
- Special Tools:**
none

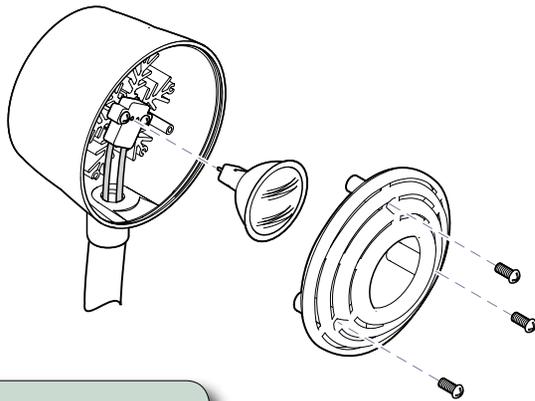


Warning
Always unplug power cord and allow light to cool.

To remove bulb...
A) Unplug light power cord.
B) Remove front bezel and bulb.

To remove lighthouse...
A) Remove hex nut, ground terminal ring, plain washer, and nylon washer.
B) Remove lighthouse from flex arm.


Note: Cut ground terminal ring, ON/OFF switch terminal connector, & bulb socket connector from wire harness.

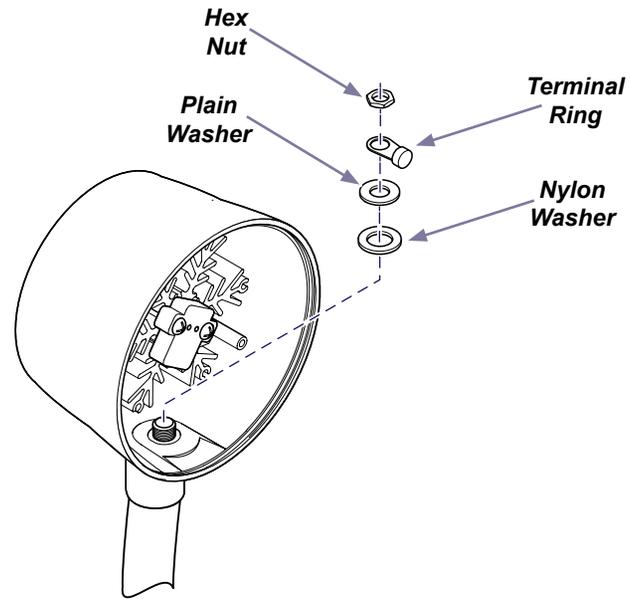

MA8338i


To remove flex arm...

- A) Slide rubber sleeve up to expose set screws.
- B) Remove two set screws & depress tabs on split retainers inward.
- C) Lift flex arm out from light post.
- D) Pull wire harness out bottom of flex arm.

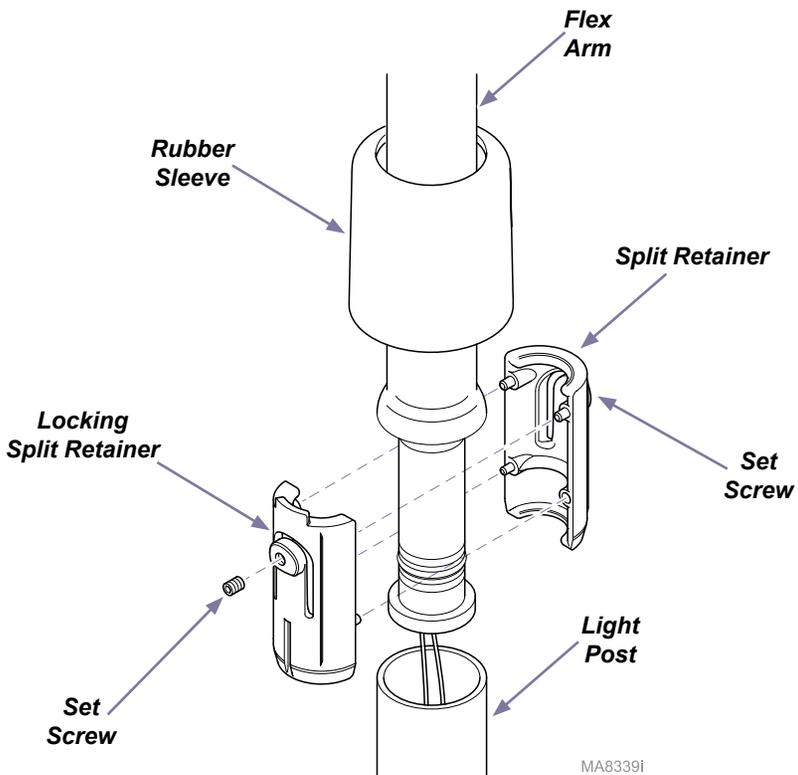

Note

There are two nylon washers supplied with kit.
Serial number prefixes NE, NC, or V use **thin** washer.
Serial number prefixes TGA or TGB use **thick** washer.

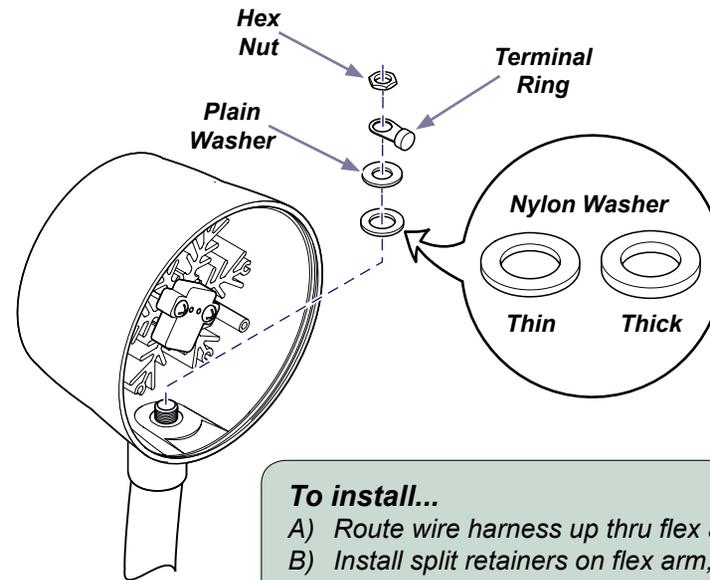

To install...

- A) Route wire harness up thru flex arm to lighthouse.
- B) Install split retainers on flex arm, slide flex arm into light post and secure with two set screws.
- C) Place lighthouse onto flex arm, crimp electrical connections and secure lighthouse with nylon washer, plain washer, ground terminal ring, and nut.

Note: Wire harness must be routed thru nylon washer, plain washer, ground terminal ring, and nut before crimping electrical connectors.

To complete...

- A) Install bulb and front bezel.
- B) Plug in power cord and check for proper operation.