



Equipment Alert

The maximum weight the shelf will support is 10 lbs (4.5 kg).



Equipment Alert

If drawer front folds past horizontal, do not operate table.

Exam Assistant® Drawer System

To use the drawer front as a shelf...

Pull drawer front down to release magnetic latch.

If drawer front folds past horizontal...

Pull drawer front upward until it snaps back into the horizontal position.



Drawer Heater Switch

To turn drawer heater ON / OFF...

Press the heater switch.

(Switch will illuminate when heater is ON)



USB

Vital Signs Devices

IQhub®...

Allows digital connection of USB and vital signs devices.

(See user's guide for details)

Maintenance

Periodically inspect the following:

- Power cord must be free of visible damage.
- All fasteners must be present and secure.
- All mechanical functions should operate
- Verify the scale is measuring accurately using a reference weight of known value.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners (see current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities). Follow this with a clear water rinse and thorough drying of material.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Calling for Service

If service is required, contact your Midmark dealer.

To contact Midmark directly:

1-800-MIDMARK (1-800-643-6275)
Monday-Friday 8:00 am until 5:00 pm (EST)



Français : Référez-vous à www.midmark.com

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
www.midmark.com



Because we care.



Quick Reference Guide 625-004 Barrier-Free® Exam Table with IQscale® and IQhub®

Model Number:
Serial Number:

Dealer Information:


Purchase Date:

Upholstery Color:

003-2445-00 Rev G (7/10/15)

This guide outlines basic operation of the table. The enclosed User Guide provides additional instructions on operation and care, as well as regulatory compliance information. If service is required, contact your authorized Midmark dealer.



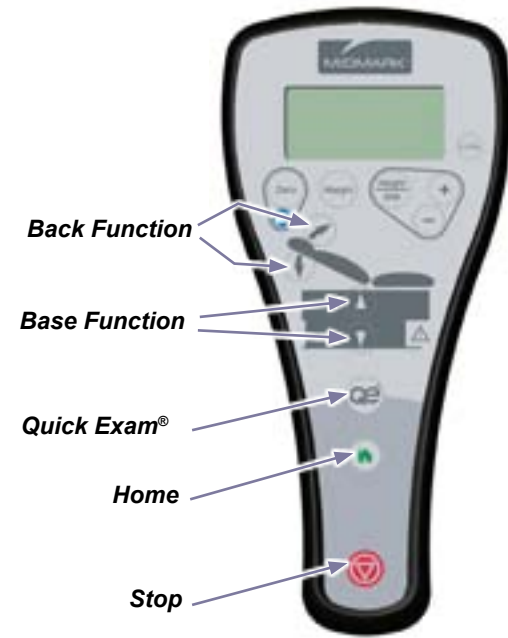

Equipment Alert

If the table's weight limit is exceeded, you will hear a "beep" and the function will be disabled. If the problem occurs with a normal load, contact your service technician.

To operate the Base / Back functions...

Press & hold the Up or Down arrow button for the desired function.

Note: The table automatically stops at its maximum and minimum height. In the event of a malfunction, press the Stop button to terminate movement.

To activate the Home -or- Quick Exam® function...

Press & release the Home -or- Quick Exam® button.

Note: In the event of a malfunction, press the Stop button to terminate movement.

To zero the IQscale®...

Before seating patient, press and release the Zero button. The display will flash "Scale Ready"; the zero operation is complete and the patient may be seated.


To measure the patient's weight...

With patient in the seated position, press & release the Weight button.

Note: To prevent the patient's feet from touching the ground, the table will automatically rise to a height of twenty five inches prior to displaying the weight.

To calculate the patient's BMI...

- a) After patient weight is displayed, press and release the Height / BMI button.
- b) Press the (+) or (-) buttons to increase or decrease patient height on display.
- c) Press the Height / BMI button again to display patient's BMI.

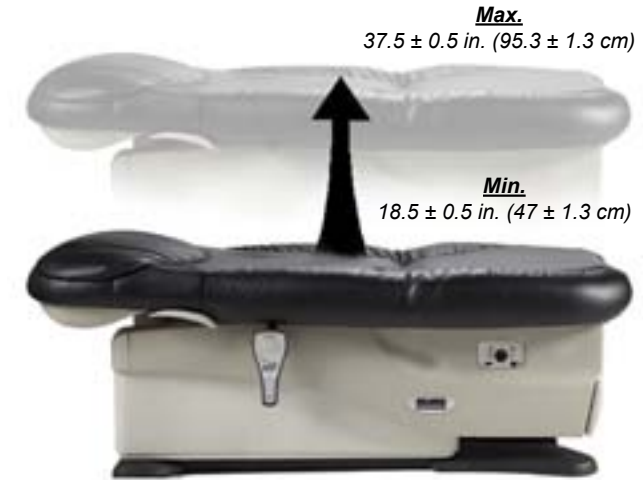

Quick Exam® Height

Programmable (set by the user)

'Home' Height

20.5 ± 0.5 in. (52.1 ± 1.3 cm)

0° (horizontal)
to +80° ±5°

To program the Quick Exam® function...

- a) Press the Base Up button to move the table to the desired position.
- b) Press & hold the Quick Exam® & Base Up buttons simultaneously for three seconds.

Note: You will hear a brief chirp when the new Quick Exam® position is saved.