


Solid State Switch Replacement [#002-1556-00]

Components:
Solid State Switch for
1.25 Motor 1

Special Tools:
none

WARNING
Equipment is not suitable for use in the presence of a flammable anesthetic mixture.

- To remove and install new solid state switch...**
- A) Disconnect power to vacuum.
 - B) Remove top cover from motor.
 - C) Remove screws from switch.
 - D) Disconnect wires from switch.
 - E) Connect wires to new switch.
 - F) Screw switch into motor.
 - G) Replace top cover on motor.
 - H) Restore power to vacuum.

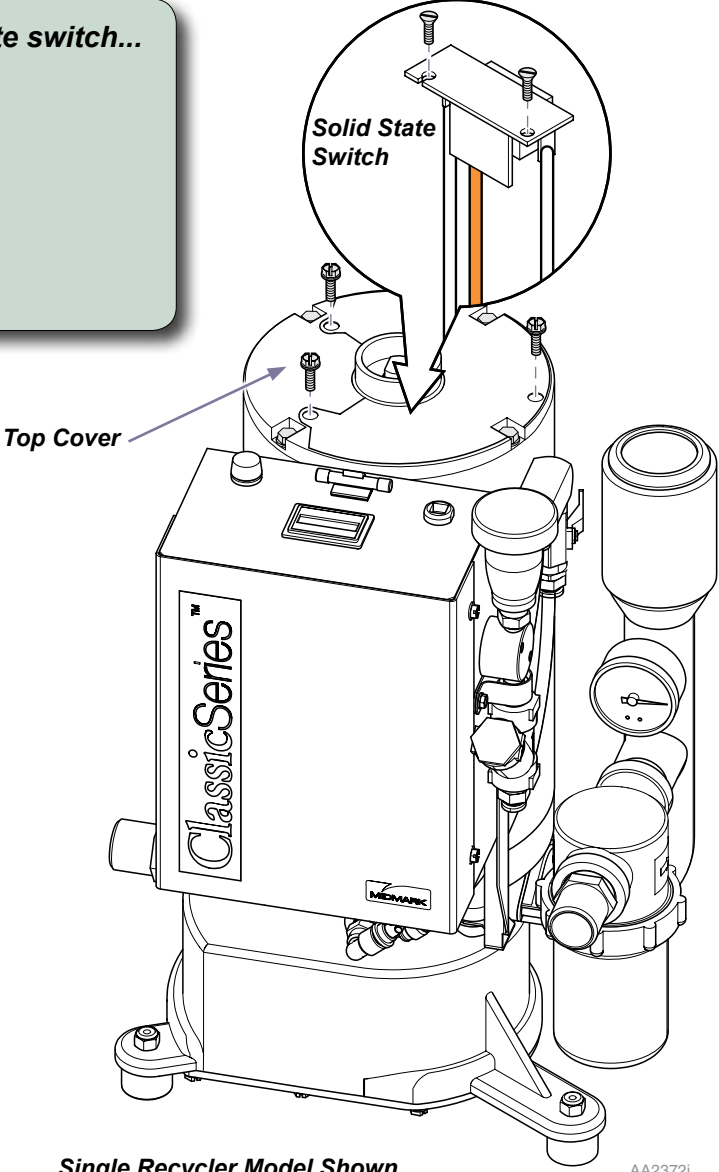

Single Recycler Model Shown AA2372i