



Wireless Basestation Kit [002-1564-00 / 002-1612-00]


Components:
PC Board..... 1

Special Tools:
none

Compatible with serial numbers:
002-1564-00 [625-001 & 625-003 (S/N: V968528 thru V1266165)]
002-1612-00 [625-001 & 625-003 (S/N: V1266166 thru Present)], [625-004 (All)]

Caution
Unplug power cord before removing stirrup guide.

Step 1: Remove two screws and stirrup guide.
Note: If necessary, remove chair arm brace.

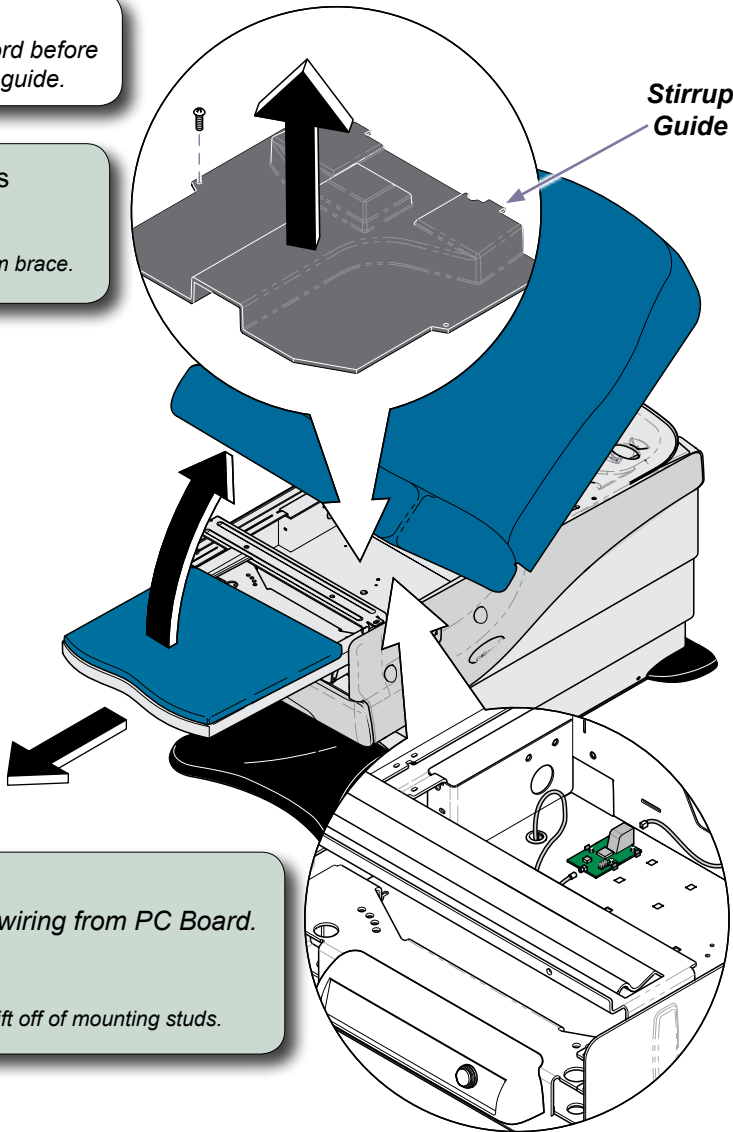

Step 3:
A) Install new PC Board.
B) Connect wiring to PC Board.
C) Install stirrup guide.
D) Perform Association Procedure.

Step 2:
A) Tag and disconnect wiring from PC Board.
B) Remove PC Board.
Tip: Push latches back, then lift off of mounting studs.

Association Procedure:
A) Disconnect power to the table for at least 3 seconds.
B*) Reconnect power, wait for the power up beep sequence from the table, then press & hold the **Stop & Home** buttons on the **wireless** controller until you hear three “beeps”.
** Note: Step B must be performed within 10 seconds of the power up beep sequence from the table.*
C) Check for proper operation of wireless controller.
D) If associating a hand **and** foot control, repeat procedure using the other wireless controller.

MA8450i