



QUICK TIPS

Using your Cardell® CMP-10 printer

- **BEFORE USING PRINTER:** Charge your printer's battery overnight using the enclosed power cord adapter.
- **BEFORE PRINTING,** disconnect the printer's power cord from printer.

Important Note: The CMP-10 printer is designed to work **ON BATTERY ONLY**. It has an internal battery that will provide over 14 hours of continuous printing.

A Suggestion: Use the printer all day on battery only, then at the end of the day, turn the printer off and reconnect the power cord to recharge the printer's battery.

Printing with your printer power cord connected will interfere with the operation of the printer and damage the battery.

- **Printer operation:**

Printer's On/Off button will switch the printer on or off only. (Use the Monitor's Print Key to begin print.) If using the serial connection, first connect the printer and monitor via the serial cable included with the printer, turn the monitor on, then turn on the printer.

Turning printer on: Push the On/Off button once to turn the printer on. A small green light will illuminate. If the printer battery needs to be charged, three short beeps will sound at startup.

Turn printer off: Hold the On/Off button for three seconds to turn the printer off. The green light will turn red, then turn back to green. Release the power button and the printer will turn off.

- **Cardell Monitor print operation:**

Initiating print: For Cardell models 9403, 9404, 9405, and 9454, hold the monitor's print key (*the small arrow pointing to the left located on the lower right corner of the monitor*) for several seconds until printing starts. The message "Printing" will appear in the Message window as a status indicator.

For the Cardell 9401 & 9402, Press and Hold the HISTORY pushbutton for two (2) seconds while in either the Event or Trend History screens. The message "Print? N" appears in the Message Window. Use either the ARROW UP or ARROW DOWN pushbutton until the message "Print? Y" appears in the Message Window. Depress the HISTORY pushbutton again to start printing the History data.

If Printer does not respond, turn it off and back on again. (*Monitor should be turned on before printer is turned on*).