


# Installing Elevance Chair Shipping Pin Kit [002-1602-00]

**Applies to Models:**  
153810 (all)  
  
Language of origin: English

**Equipment Alert**  
Do not relocate chair without shipping pin.

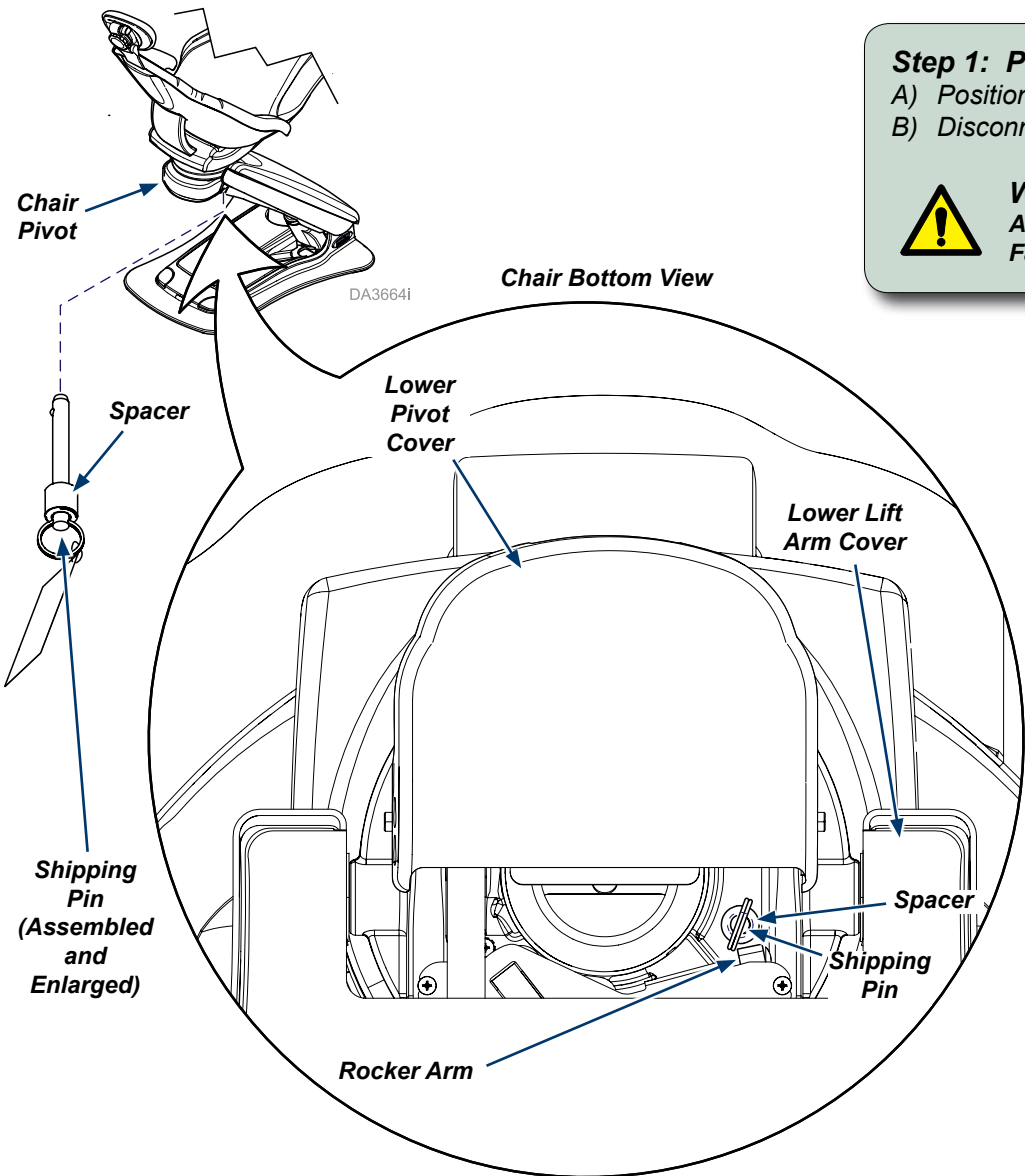

**Step 1: Position chair.**  
A) Position chair so lower lift arm cover position allows adequate space for the installation.  
B) Disconnect power from the chair.

**WARNING**  
Always disconnect power from the chair before performing installations. Failure to do so may result in personal injury.

**Step 2: Install shipping pin.**  
A) Secure tag to the pin.  
B) Install spacer to shipping pin (position near pin ring).  
C) Install the shipping pin fully into the pivot while moving the rocker arm as needed.

**Note:** The shipping pin prevents the chair from pivoting during transportation and shipment.

D) Position spacer against pivot casting.  
E) Connect the power to the chair and verify operations.  
F) Position chair at lowest height before transportation or shipment.

TP200 Rev. A