
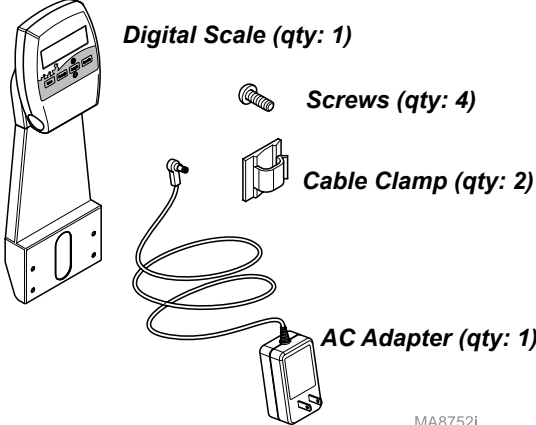
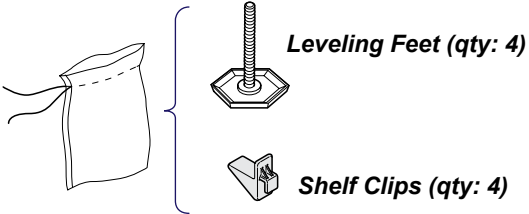
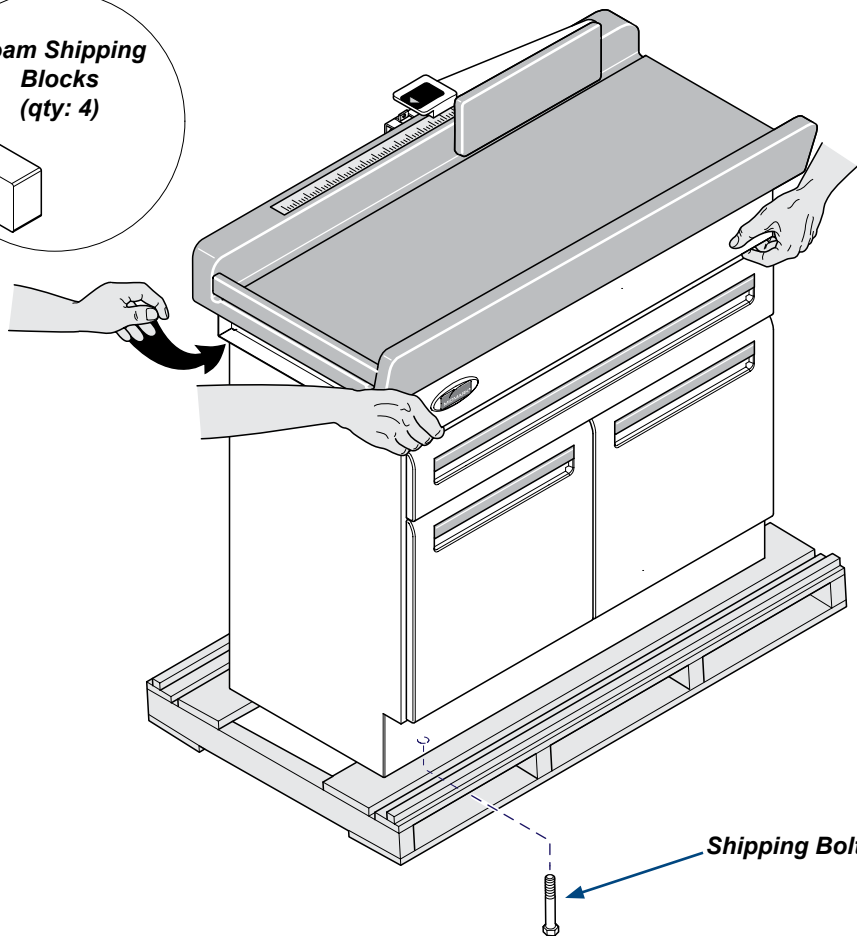
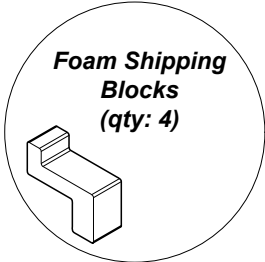
Pediatric Exam Table Installation Guide

Applies to Models:
640

Step 1: Remove table from skid.
 A) Remove three shipping bolts.
 B) Remove four foam shipping blocks from corners under the upholstered top.
 C) Open drawer and carefully lift table off of skid.
 D) Remove shipping carton from bottom of table.
 Note: See illustration for proper lifting points.

Equipment Alert
 Do not lift table by grasping the upholstered top. Refer to illustration for proper lifting points.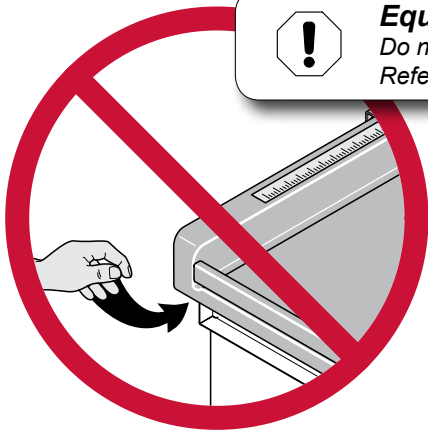


MA8752I

Step 2: Install digital scale.

- A) Secure scale tower to back of table with four screws.
- B) Verify there is clearance between the tower and the scale table.
- C) Connect scale cable to digital display cable.
- D) Snap cable into cable clamp.


Note: Align cable arrows as shown for proper connection.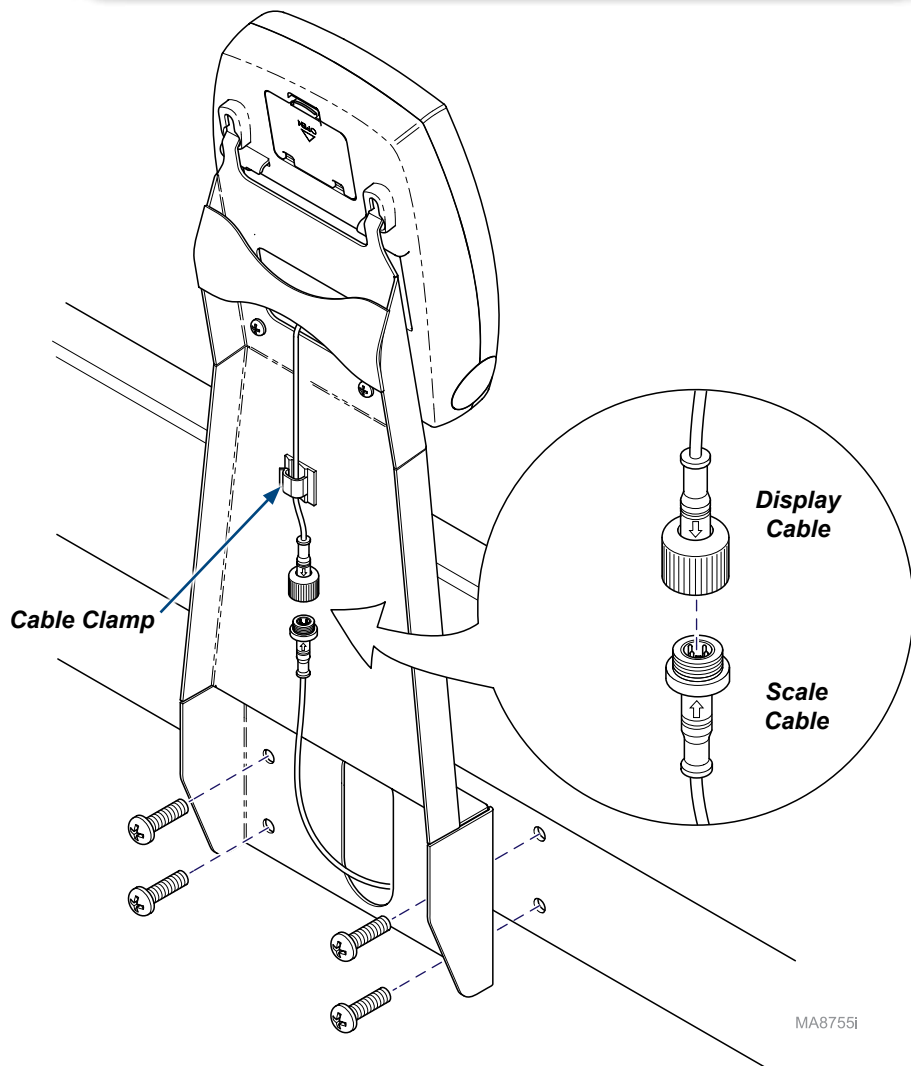

Step 3: Connect power to the digital scale.

- A) Remove access cover.
- B) Route AC Adapter cable behind tower shield.
- C) Connect AC Adapter to digital scale.
- D) Reinstall access cover.

Electrical Requirements
120 VAC, 60 Hz, 5W
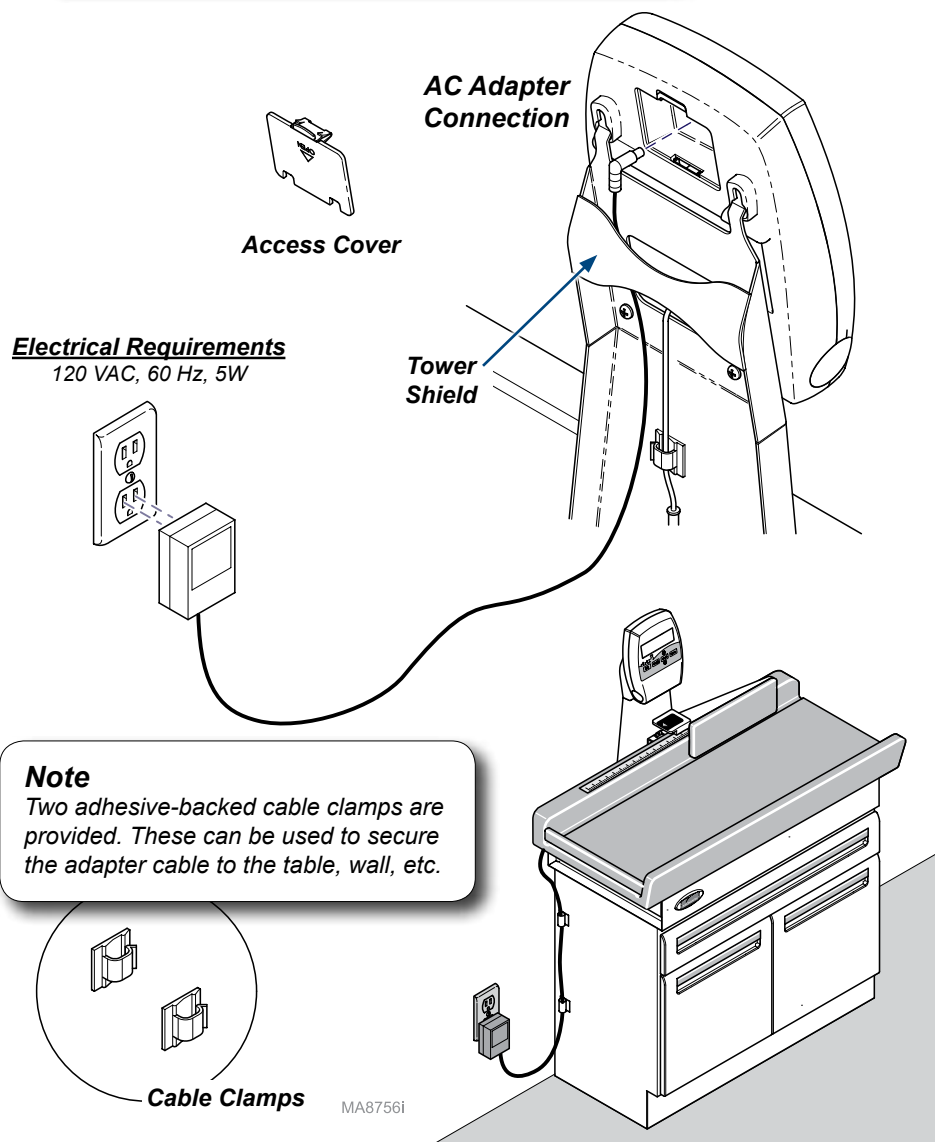


**Equipment Alert**

To ensure proper operation, the table must be level.

Step 4: Level the table.

- A) Install four leveling feet.
- B) Place a level on table top.
- C) Adjust leveling feet until table is level.

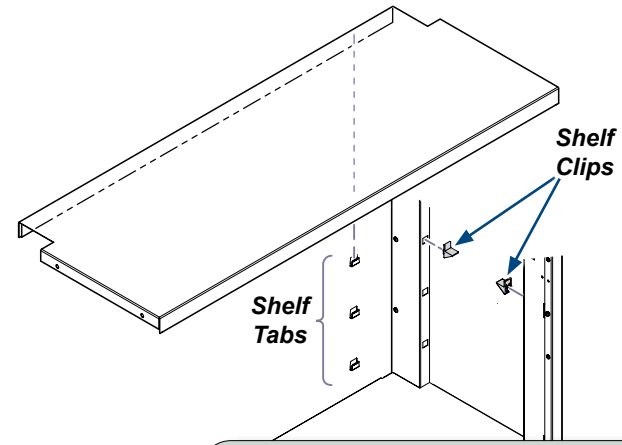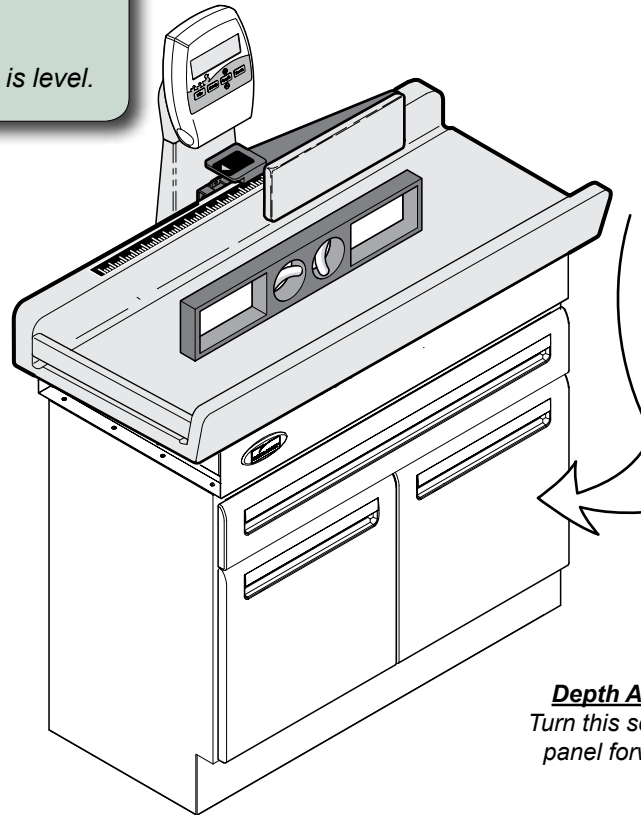**Step 5: Install storage shelf.**

- A) Install four shelf clips at desired height.
- B) Latch rear lip of shelf into shelf tabs.
- C) Lower shelf onto shelf clips.

Depth Adjustment
Turn this screw to move panel forward / back.

Vertical Adjustment
Loosen this screw on both hinges.
Move door up / down, then tighten screws.

Horizontal Adjustment
Turn this screw to move panel side to side.

Step 6: Adjust doors if needed.**Leveling Feet**

Qty: 4 (located at each corner)

**Equipment Alert**

Do not place side of exam table against wall. It could result in scale inaccuracies.

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