



## Traction Mat [9A549001]


### Applies to Models:

8012\*  
8012-00-001  
8012-00-XXX

Language of origin: English

### Note

The traction mat should be cleaned between patients with a lint free cloth, dampened with warm water and a mild detergent or a 10% solution of bleach with 90% water (use only 5.25% bleach). Dry with a clean cloth or paper towel. The mat can be moved to a tub or sink for a more thorough cleaning.

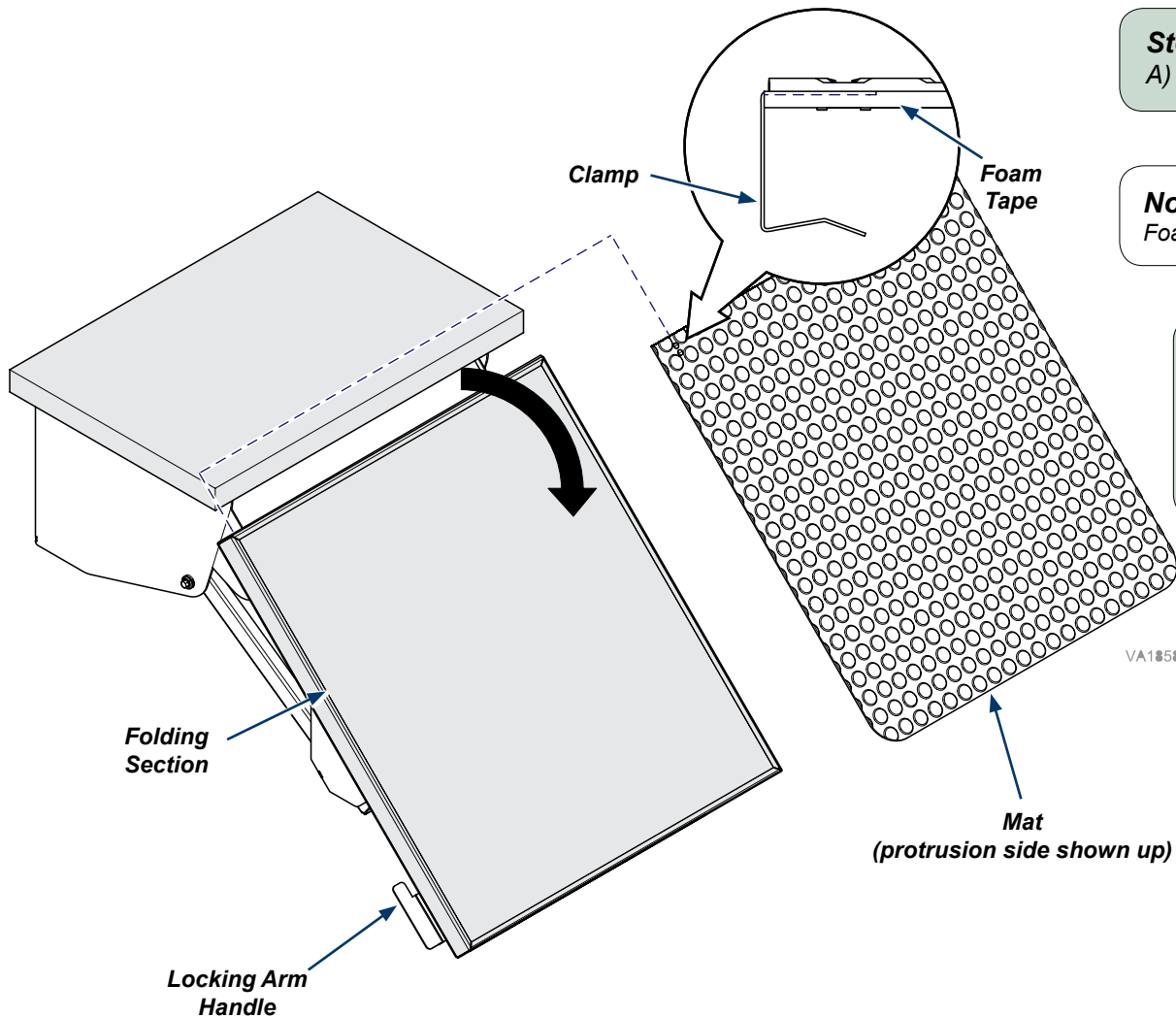

### Step 1: Unroll mat.

A) Straighten mat out by rolling in opposite direction of curl.

### Note

Foam tape is required to protect table surface and to provide a snug fit.

### Step 2: Lower table folding section.

- A) Hold folding section then lift one side of the locking arm handle.
- B) Move folding section down allowing enough room for mat installation.

### Step 3: Install mat.

- A) Install mat to folding section with protrusion side up.
- B) Move folding section in original position.
- C) Ensure mat correctly aligns with table.