



IQvitals® to IQscale™ Configuration

Applies to:

9A478
9A551001
9A629001
9A629002



WARNING

IEC 60950 approved Information Technology Equipment must be placed outside the "patient environment." Refer to safety instructions in the exam table and IQvitals® operation manuals.

Note

This document is provided as a brief overview of steps to follow when configuring the IQvitals® device to communicate with the IQscale™. For detailed information regarding the operation and functions of the IQvitals® device refer to the IQvitals® operation manual.

To configure...

- A) Press **POWER BUTTON** to turn on the IQvitals® device.
- B) Press the **SETTINGS** icon to display the password screen.
- C) Enter the default password of **986**.
- D) Press **OK** to display the settings screen.
- E) Press **MONITOR SETTINGS**.
- F) Press the **MORE** button.
- G) Press the **TABLE** button.
- H) Select **YES** for table connection.
- I) Press **OK**.
- J) Press the **HOME** icon to return to the main screen. The **TABLE** icon should be visible on the main screen. If the **TABLE** icon is not visible, confirm the monitor settings by repeating the steps above.