

# 255 LED Light Module Installation [002-1892-00]

**Components:**  
Light Module

**Special Tools:**  
none

Language of origin: English

**Warning**  
Always disconnect power before removing covers. Failure to do so may result in personal injury.

**Caution**

- Carefully handle parts near membranes and wire harnesses.
- Allow the light to cool before removing.




Carefully move the light away from the exam site to prevent contaminants from falling into the area.

**Note**  
Use only factory replacement lights (refer to parts list for order number).

**Warning**  
Always disconnect power cord and allow time for light to cool before replacing. Do not touch the cone section of the LED light or reflector in light housing.

**To remove light...**

- Turn OFF facility power breaker so there is no power in customer supplied wiring.
- Turn light switch OFF (⏻).
- Remove screws and screw caps from reflector.
- Separate front ring and faceplate from reflector.
- Disconnect LED light module connector from LED light module.
- Remove screws from LED light module.

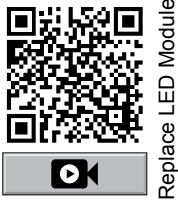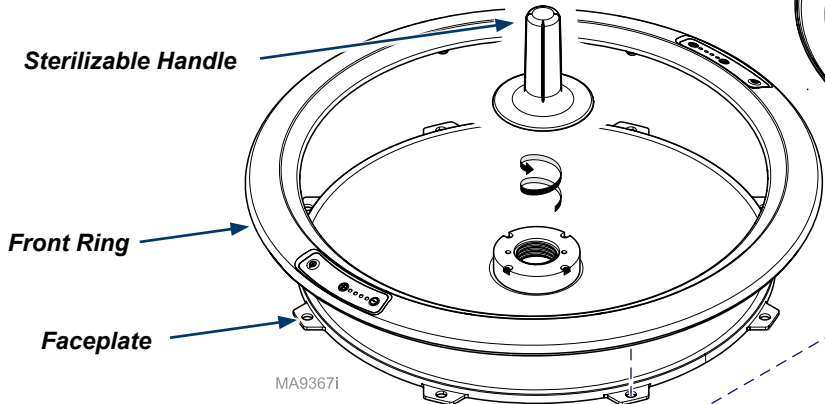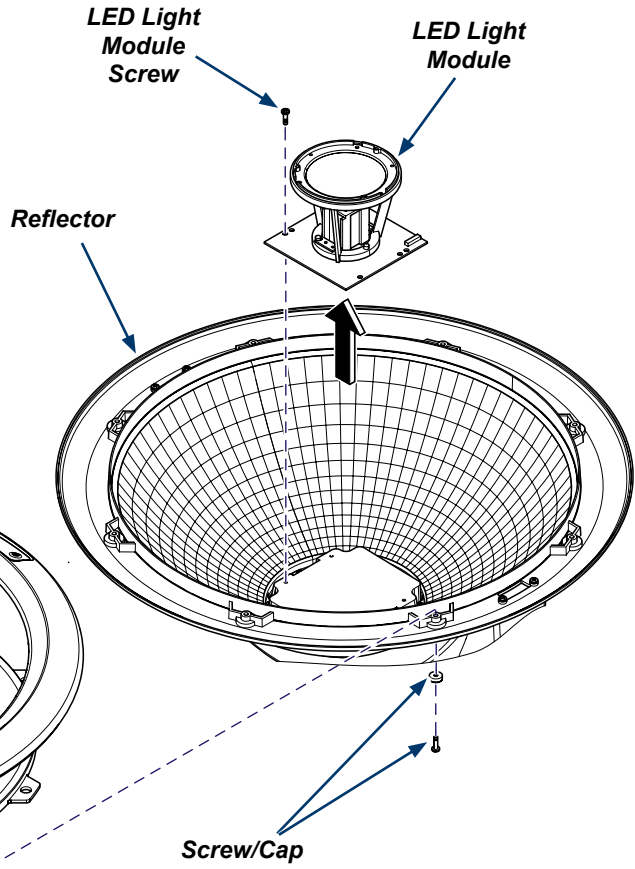

Replace LED Module

TP200 Rev. A



**Caution**

- Carefully handle parts near membranes and wire harnesses.
- Carefully move the light away from the exam site to prevent contaminants from falling into the area.



**Equipment Alert**

Do not over-tighten screws.




**Warning**

Do not touch the cone section of the LED light or reflector in light housing.

**To install light...**

- Peel film paper off new LED light module then install into previous place holding LED light module with a clean cloth.
- Complete Step 1, substeps B through E in reverse order.
- Connect power and verify operation.

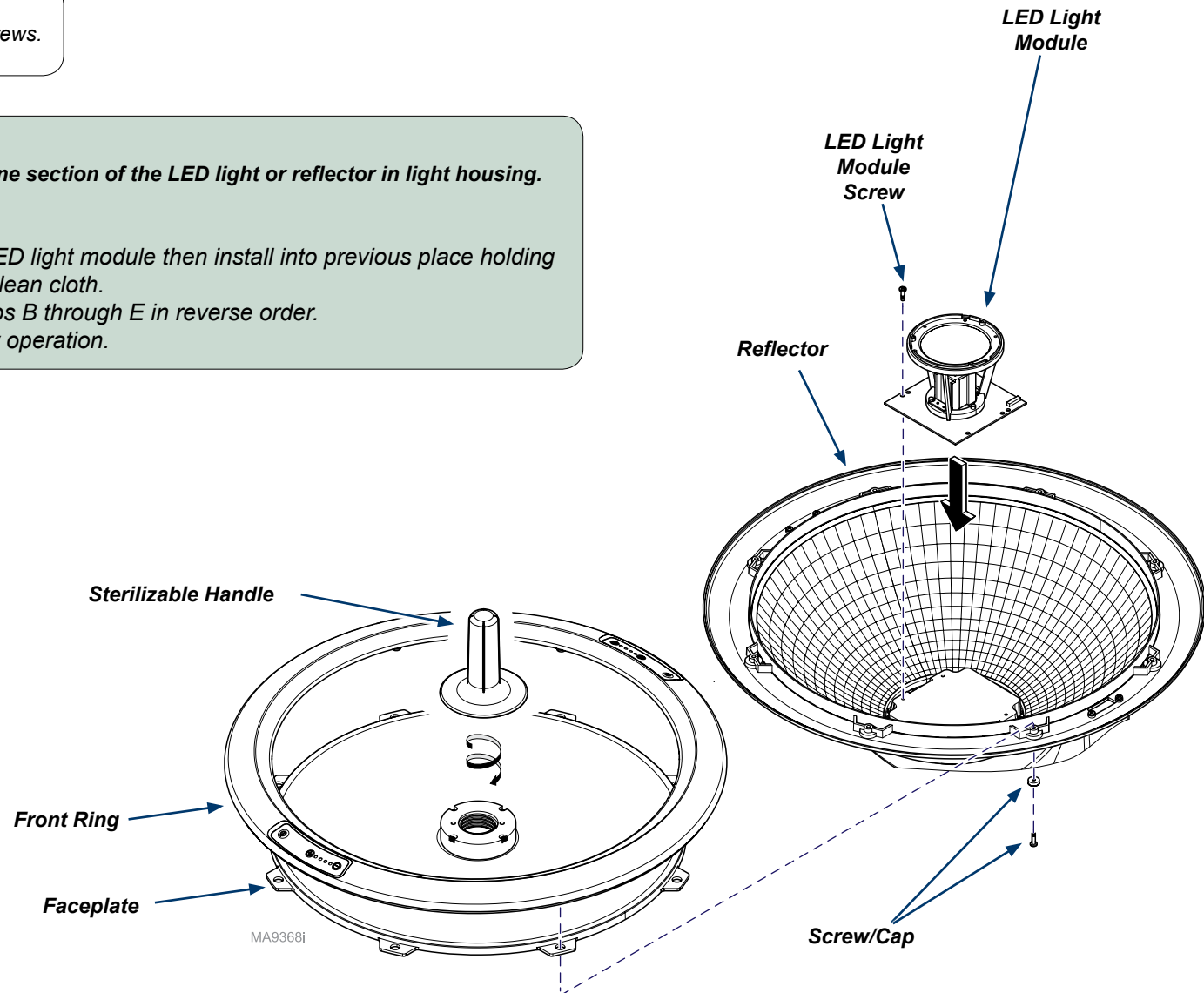

Because we care.