

255 LED Light Module Installation [002-1892-00]

Components:

Light Module

Special Tools:

none

Language of origin: English



Warning

Always disconnect power before removing covers.
Failure to do so may result in personal injury.



Caution

- Carefully handle parts near membranes and wire harnesses.
- Allow the light to cool before removing.
- Carefully move the light away from the exam site to prevent contaminants from falling into the area.

Note

Use only factory replacement lights (refer to parts list for order number).



Warning


Always disconnect power cord and allow time for light to cool before replacing.
Do not touch the cone section of the LED light or reflector in light housing.

To remove light...

- Turn OFF facility power breaker so there is no power in customer supplied wiring.
- Turn light switch OFF (⏻).
- Remove screws and screw caps from reflector.
- Separate front ring and faceplate from reflector.
- Disconnect LED light module connector from LED light module.
- Remove screws from LED light module.



Replace LED Module

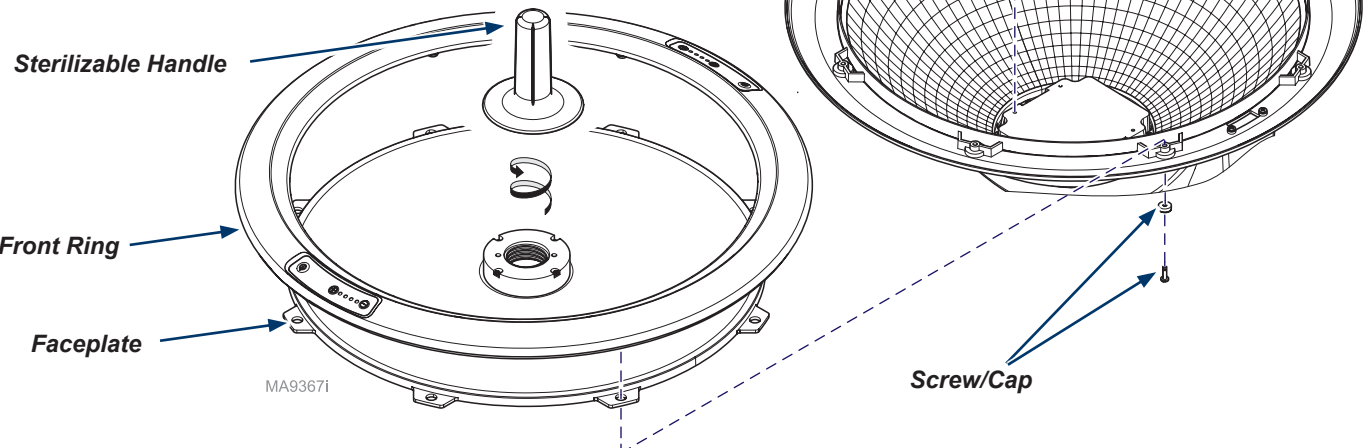

**Caution**

- Carefully handle parts near membranes and wire harnesses.
- Carefully move the light away from the exam site to prevent contaminants from falling into the area.

**Equipment Alert**

Do not over-tighten screws.

**Warning**

Do not touch the cone section of the LED light or reflector in light housing.

To install light...

- Peel film paper off new LED light module then install into previous place holding LED light module with a clean cloth.
- Complete Step 1, substeps B through E in reverse order.
- Connect power and verify operation.

