


Single Articulating Headrest

Note: This model uses Pebble Grey painted components

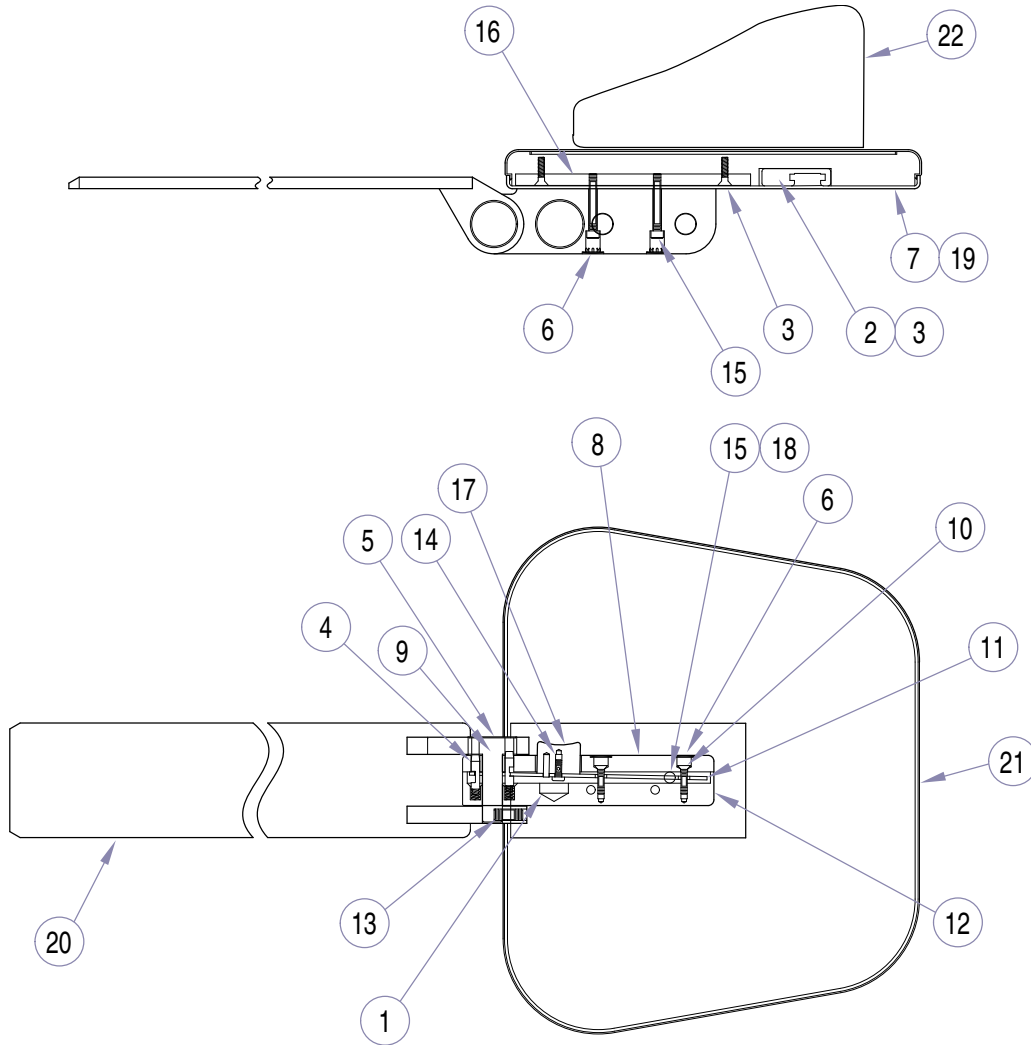

MA613400i

Item	Part No.	Description	Qty.	Item	Part No.	Description	Qty.
	154078-xxx	Single Art. Headrest (includes items 1 thru 22)	1	12	• 122777-50	• Spring Housing	1
1	• 103314	• Spring	4	13	• 122778	• Screw (use adhesive #042-0024-00) ...	1
2	• 121493-01	• Headrest Receptacle	1	14	• 122784	• Screw	1
3	• 121784	• Screw	6	15	• 122775	• Release Plate Rocker	2
4	• 121918-3	• Stop Pin	4	16	• 122805-50	• Mounting Plate	1
5	• 121945	• Decorative Disc	2	17	• 153258-50	• Push Button Assembly	1
6	• 121968-50	• Plug	4	18	• 185171	• Lubricant	AR
7	• 122677-01	• Back Cover	1	19	• 275184	• Foam Tape	AR
8	• 122757-50	• Pin Housing	1	20	• 153249-50	• Tang Weldment	1
9	• 122772-3	• Pivot Pin	1	21	• 154090-xxx	• Headrest Board (vacu-form)	1
10	• 122096	• Screw	2	22	• 154092-xxx	• Headrest Board (plush)	1
11	• 122776-2	• Release Plate	1		• 154089-xxx	• Magnetic Cushion (vacu-form)	1
					• 154091-xxx	• Magnetic Cushion (plush)	1

Always Specify Model & Serial Number