


# Double Articulating Headrest

Note: This model uses Pebble Grey painted components

Order   
Medical and  
Animal Health  
Parts Online

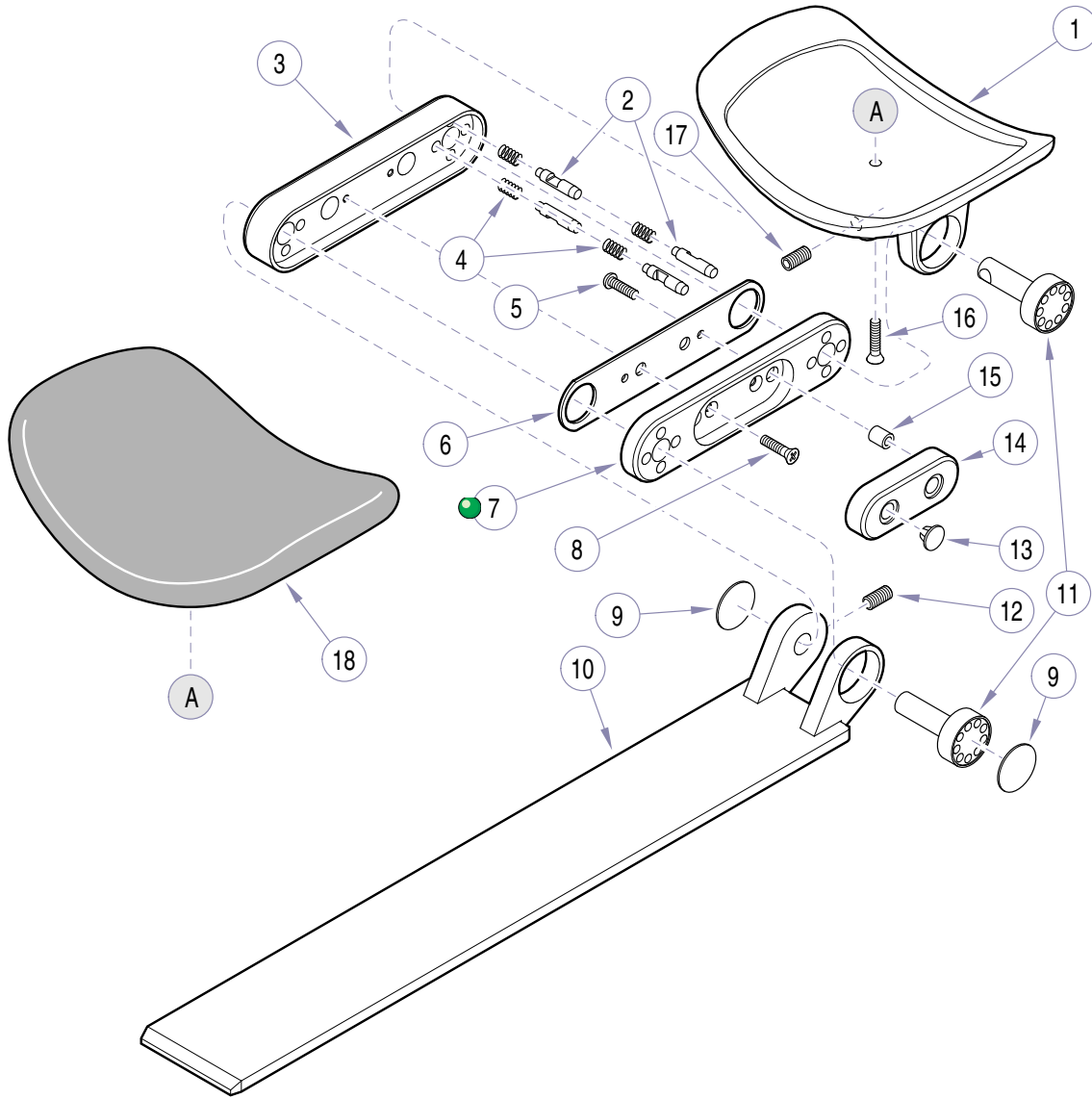

MA543500

| Item | Part No.          | Description                      | Qty. | Item | Part No.     | Description                  | Qty. |
|------|-------------------|----------------------------------|------|------|--------------|------------------------------|------|
|      | <b>154087-xxx</b> | <b>DbI Art. Headrest</b>         |      |      |              |                              |      |
|      |                   | (includes items 1 thru 18) ..... | 1    |      |              |                              |      |
| 1    | • 121911-00       | • Headrest Housing .....         | 1    | 10   | • 152905-50  | • Tang Assembly .....        | 1    |
| 2    | • 121918-3        | • Stop Pin .....                 | 8    | 11   | • 122772-3   | • Pivot Pin .....            | 2    |
| 3    | • 121913-50       | • Spring Housing .....           | 1    | 12   | • 122778     | • Set Screw .....            | 1    |
| 4    | • 103314          | • Spring .....                   | 8    | 13   | • 121968-00  | • Plug .....                 | 2    |
| 5    | • 122097          | • Screw .....                    | 2    | 14   | • 121915-50  | • Button .....               | 1    |
| 6    | • 121917-2        | • Release Plate .....            | 1    | 15   | • 122065-2   | • Spacer .....               | 2    |
| 7    | • 121912-50       | • Pin Housing .....              | 1    | 16   | • 122089-50  | • Screw .....                | 1    |
| 8    | • 122096          | • Screw .....                    | 2    | 17   | • 121488     | • Set Screw .....            | 1    |
| 9    | • 121945          | • Decorative Disc .....          | 4    | 18   | • 154079-xxx | • Vacu-Form Upholstery ..... | 1    |
|      |                   |                                  |      |      | • 154080-xxx | • Plush Upholstery .....     | 1    |

Always Specify Model & Serial Number