



Wiring Diagram: DC Power Unit

Item	Description	Part Number
A	Fan Extension Harness	301-1005-WH-004
B	Battery #1 Temp. Sensor Harness	304-1001-AS-001
C	Battery #2 Temp. Sensor Harness	304-1001-AS-002
D	CAT 5 Cable	ACC-0110
E	USB Harness	CBL-0046
F	Red/Black Battery Harness (qty: 2)	305-1001-WH-008
G	Computer Adapter Harness	-
H	Monitor Adapter Harness (optional)	-

[Wire Color Abbreviation Chart](#)

Page Applies To:	Model Numbers:	Build Dates:
Care Exchange DC Carts	6251 / 6252 / 6256	all

TP204 Rev. A

Wiring Color Abbreviation Chart

Item	Color (English)	Farbe (Deutsch)	Couleur (Français - Europe)	Color (español europeo)	Colore (italiano)	Color (others)
BK	Black	Schwarz	Noir	Negro	Nero	
BL	Blue	Blau	Bleu	Azul	Blu	
BR	Brown	Braun	Marron	Marrón	Marrone	
CL	Clear	Durchsichtig	Incolore	Transparente	Trasparente	
GN	Green	Grün	Vert	Verde	Verde	
GY	Grey	Grau	Gris	Gris	Grigio	
OR	Orange	Orange	Orange	Naranja	Arancione	
PR	Purple	Purpur	Pourpre	Morado	Porpora	
CLPR	Clear Purple	Durchsichtig Purpur	Incolore Pourpre	Transparente Morado	Trasparente Porpora	
RD	Red	Rot	Rouge	Rojo	Rosso	
WH	White	Weiß	Blanc	Blanco	Bianco	
YL	Yellow	Gelb	Jaune	Amarillo	Giallo	
PK	Pink	Pink	Rose	Rosa	Rosa	
TN	Tan	Hellbraun	Fauve	Marrón claro	Tanè (marrone scuro)	
BK/WH	Black/White	Schwarz / Weiß	Noir / Blanc	Negro / Blanco	Bianco / Nero	
BL/WH	Blue/White	Blau / Weiß	Bleu / Blanc	Azul / Blanco	Blu / Bianco	
BR/WH	Brown/White	Braun / Weiß	Marron / Blanc	Marrón / Blanco	Marrone / Bianco	
CL/BK	Clear / Black	Durchsichtig / Schwarz	Incolore / Noir	Transparente / Negro	Trasparente / Nero	
GN/WH	Green / White	Grün / Weiß	Vert / Blanc	Verde / Blanco	Verde / Bianco	
GN/YL	Green / Yellow	Grün / Gelb	Vert / Jaune	Verde / Amarillo	Verde / Giallo	
OR/WH	Orange / White	Orange / Weiß	Orange / Blanc	Naranja / Blanco	Arancione / Bianco	
PR/WH	Purple / White	Purpur / Weiß	Pourpre / Blanc	Morado / Blanco	Porpora / Bianco	
RD/WH	Red / White	Rot / Weiß	Rouge / Blanc	Rojo / Blanco	Rosso / Bianco	
VI/WH	Violet / White	Violett / Weiß	Violet / Blanc	Violeta / Blanco	Viola / Bianco	
YL/WH	Yellow / White	Gelb / Weiß	Jaune / Blanc	Amarillo / Blanco	Giallo / Bianco	
YL/GN	Yellow / Green	Gelb / Grün	Jaune / Vert	Amarillo / Verde	Giallo / Verde	
YL/BK	Yellow / Black	Gelb / Schwarz	Jaune / Noir	Amarillo / Negro	Giallo / Nero	