


Wiring Diagram: AC Power Unit (with Sealed Lead Acid Battery)

Item	Description	Part Number
A	AC IN (part of AC Controller)	-
B	AC OUT (part of AC Controller)	-
C	Battery - 12V, 20A Max (part of AC Controller)	-
D	Battery Lead Harness (Red/Black)	-
E	COMM 1 - Grey (RUI communication Port)	AS-1306-2
F	COMM 2 - Black (RUI communication Port)	AS-1306-1
G	COMM 3 - Yellow (Battery L-ION Port)	-
H	USB- Software Communication Port	AS-1308
I	3-Outlet Power Cord	AS-1302
J	Power Extension Cord	AS-1304
K	Coiled Power Cord	AS-617


Page Applies To:	Model Numbers:	Serial Numbers:
Care Exchange AC Carts (with Sealed Lead Acid Battery)	6261 / 6263	Prior to 87023-06R-1530042

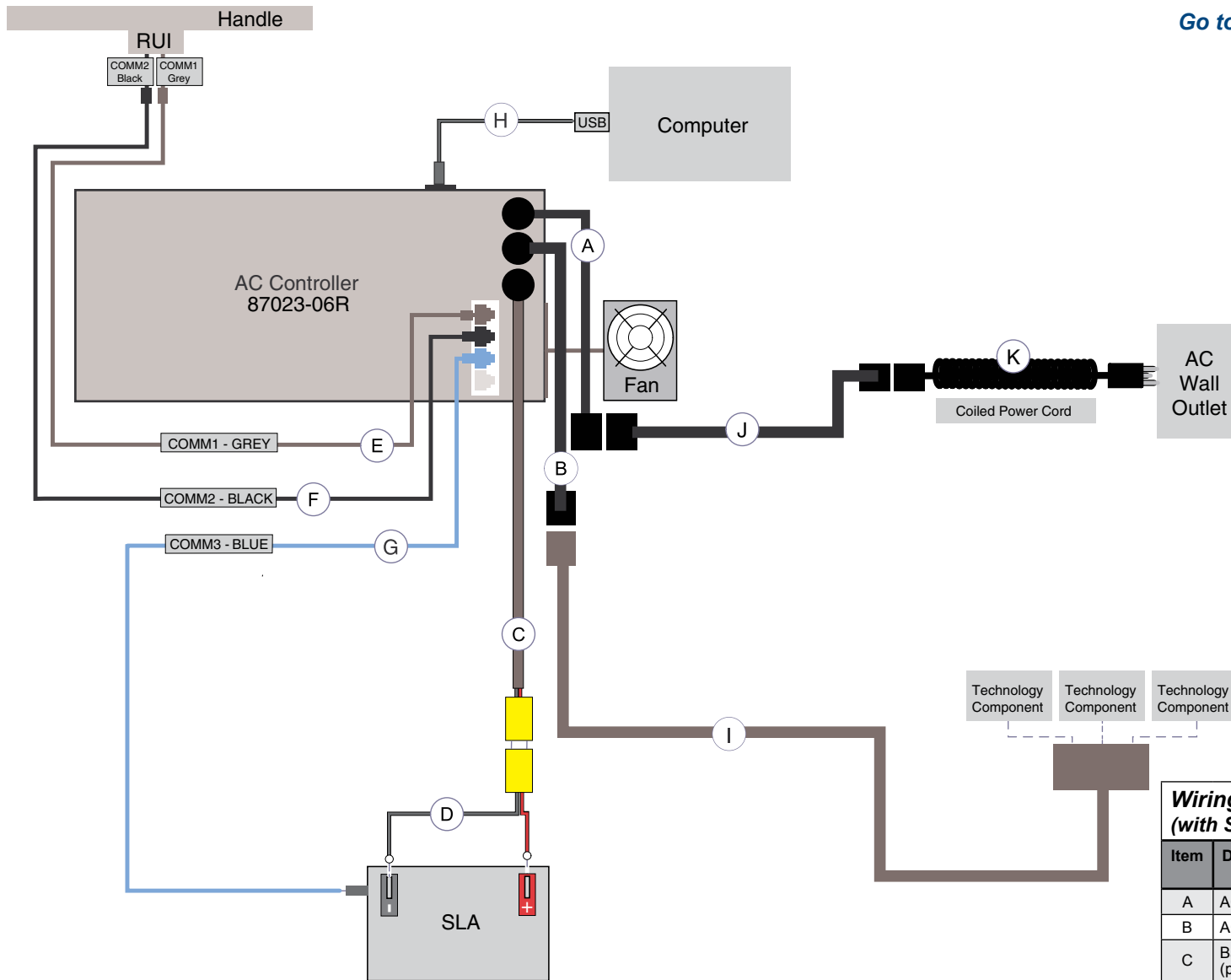

Wiring Diagram: AC Power Unit (with Sealed Lead Acid Battery)

Item	Description	Part Number
A	AC IN (part of AC Controller)	-
B	AC OUT (part of AC Controller)	-
C	Battery - 12V, 20A Max (part of AC Controller)	-
D	Battery Lead Harness (Red/Black)	-
E	COMM 1 - Grey (RUI communication Port)	AS-1306-2
F	COMM 2 - Black (RUI communication Port)	AS-1306-1
G	COMM 3 - Yellow (Battery L-ION Port)	-
H	USB- Software Communication Port	AS-1308
I	3-Outlet Power Cord	AS-1302
J	Power Extension Cord	AS-1304
K	Coiled Power Cord	AS-617

Page Applies To:	Model Numbers:	Serial Numbers:
Care Exchange AC Carts (with Sealed Lead Acid Battery)	6261 / 6263	87023-06R-1530042 thru Present

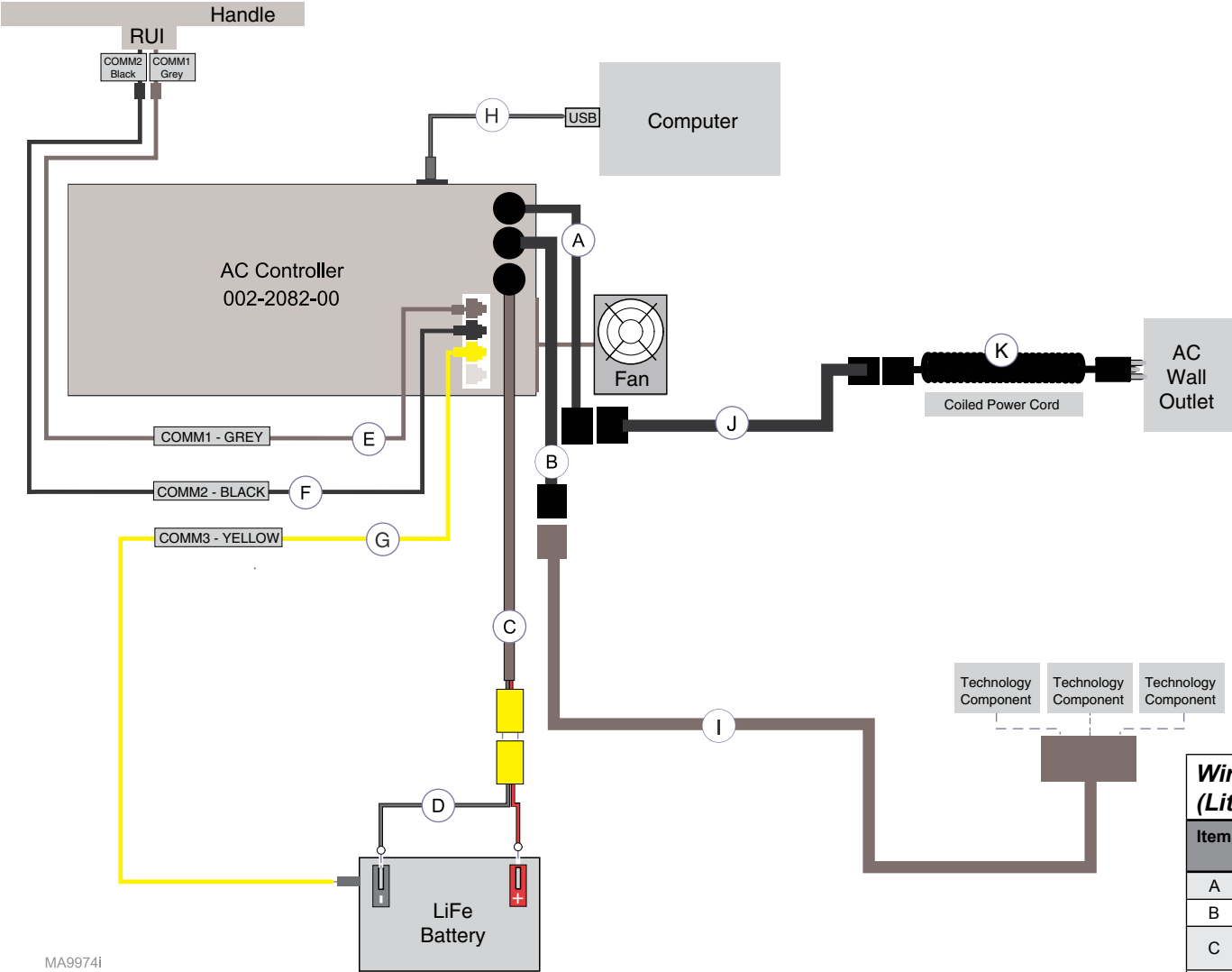


MA9974i

Wiring Diagram: AC Power Unit (Lithium Battery)

Item	Description	Part Number
A	AC IN (part of AC Controller)	-
B	AC OUT (part of AC Controller)	-
C	Battery - 12V, 20A Max (part of AC Controller)	-
D	Battery Lead Harness (Red/Black) (part of Battery Pack)	-
E	COMM 1 - Grey (RUI communication Port)	AS-1306-2
F	COMM 2 - Black (RUI communication Port)	AS-1306-1
G	COMM 3 - Yellow (Battery L-ION Port)	-
H	USB- Software Communication Port	AS-1308
I	3-Outlet Power Cord	AS-1302
J	Power Extension Cord	AS-1304
K	Coiled Power Cord	AS-617

Page Applies To:	Model Numbers:	Serial Numbers:
Care Exchange AC Carts (with Lithium Battery)	6261 / 6263	Prior to 87023-06R-1530042

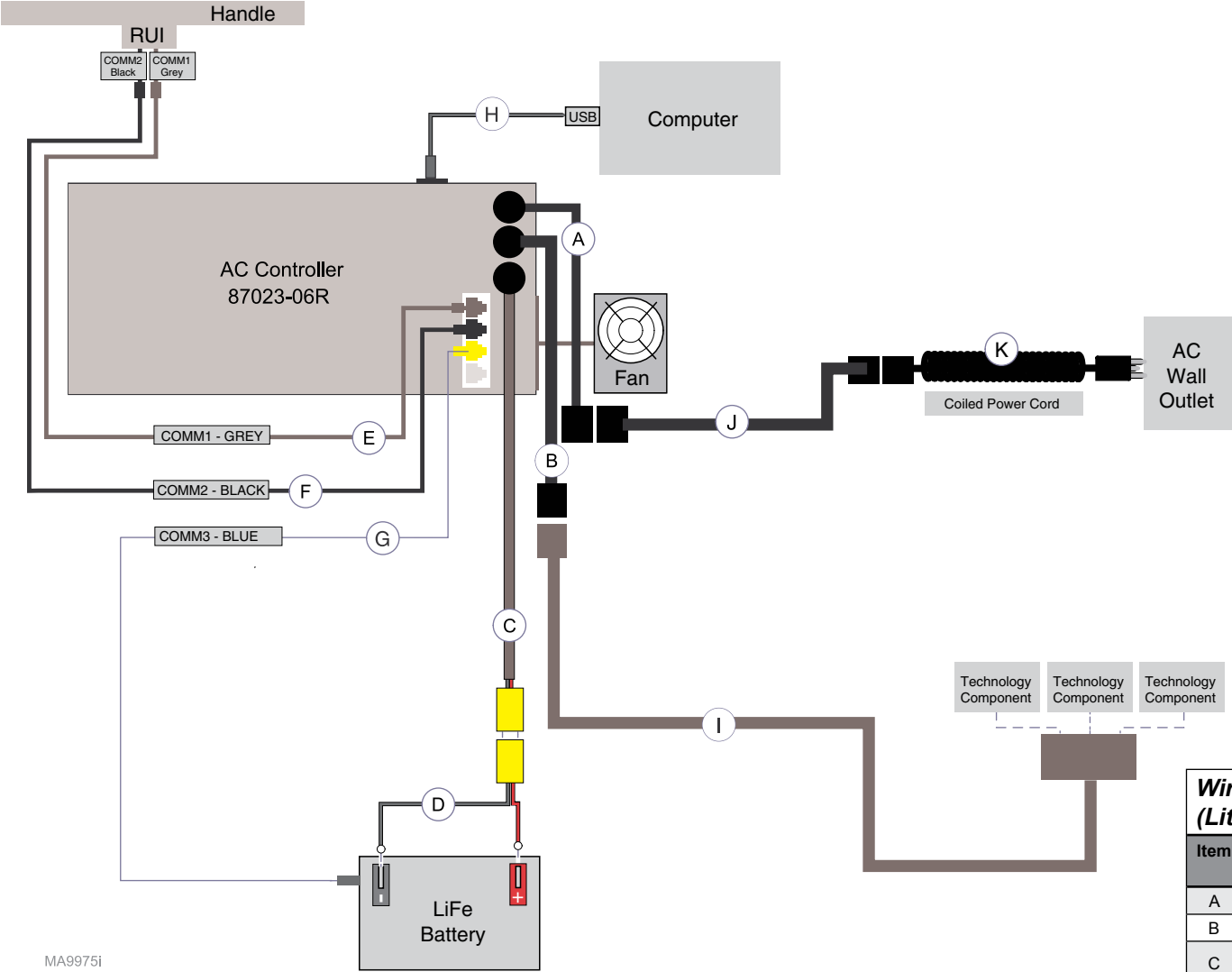

MA9975i

Wiring Diagram: AC Power Unit (Lithium Battery)

Item	Description	Part Number
A	AC IN (part of AC Controller)	-
B	AC OUT (part of AC Controller)	-
C	Battery - 12V, 20A Max (part of AC Controller)	-
D	Battery Lead Harness (Red/Black) (part of Battery Pack)	-
E	COMM 1 - Grey (RUI communication Port)	AS-1306-2
F	COMM 2 - Black (RUI communication Port)	AS-1306-1
G	COMM 3 - Blue (Battery L-ION Port)	-
H	USB- Software Communication Port	AS-1308
I	3-Outlet Power Cord	AS-1302
J	Power Extension Cord	AS-1304
K	Coiled Power Cord	AS-617

Page Applies To:	Model Numbers:	Serial Numbers:
Care Exchange AC Carts (with Lithium Battery)	6261 / 6263	87023-06R-1530042 thru Present