

# Suggested Premixed Disinfectants for Upholstery

## Notes

Midmark's standard policy on disinfecting medical equipment, which is printed in the Care and Maintenance portion of the User Guides, is to use a solution of bleach and water mixed in a 1:10 ratio (10%) to disinfect the upholstery.<sup>1</sup> This should be followed with a clear water rinse and thorough drying of material. Any deviation from this recommendation could risk damage to the upholstery, possibly causing premature cracking, staining or discoloration.

For convenience you may wish to purchase a premixed disinfecting solution in spray or wipe form. Bleach based products manufactured with a 1 in 10 ratio (10%) have been shown to be compatible. The following off the shelf disinfecting solutions have been tested by an independent lab and deemed safe for use on our upholstery and all surfaces of our exam and procedure tables. This list supersedes all previous versions (before April 2018).

## Here are some available premixes:



- Clorox Healthcare™ Bleach Germicidal Wipes**  
 EPA Reg. No 67619-12  
 6" x 5" Clinical Wipes, Item No. 30577  
 6.75" x 9" Multipurpose Wipes, Item No. 35309  
 12" x 12" Terminal Wipes, Item No. 30358  
 12" x 12" Terminal Wipes Refill, Item No. 30359



- Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant**  
 EPA Reg. No 67619-25  
 6.75" x 9" Wipes, Item No. 30824  
 6.75" x 5.75" Wipes, Item No. 30825  
 12" x 11" Wipes, Item No. 30826  
 12" x 11" Wipes Refill, Item No. 30827  
 EPA Reg. No 67619-24  
 32 ounce Spray, Item No. 30828  
 32 ounce Spray Refill, Item No. 30829



- PDI Sani-Cloth® Plus Germicidal Disposable Cloth**  
 EPA Reg. No 9480-6  
 6" x 6.75" Wipes, Large Canister (160 count) Item No. Q89072  
 7.5" x 15" Wipes, X-Large Canister (65 count) Item No. Q85084



- Metrex™ CaviWipes®**  
 EPA Reg. No 46781-8  
 6" x 6.75" CaviWipes, (160 count) Item No. 13-1100  
 9" x 12" CaviWipesXL, (65 count) Item No. 13-1150

**List continues.**

### **More available premixes:**



- **Alcavis® Bleach-Wipes**  
EPA Reg. No 65787-2  
8" x 10" Alcavis Bleach-Wipes, (100 count) Item No. 09041-100



- **PURELL® Healthcare Surface Disinfectant**  
EPA Reg. No 84368-1-84150  
32 fl. oz. Spray 6 pack, Item No. 3340-06  
1 gal. Spray Refill 4 pack, Item No. 4340-04



- **Clorox Healthcare® VersaSure® Cleaner Disinfectant Wipes**  
EPA Reg. No 67619-37  
85 ct. canister (6.75" x 8") Item No. 31757  
150 ct. canister (6" x 5") Item No. 31758  
110 ct. bucket (12" x 12") Item No. 31759  
30 ct. pouch (6.75" x 8") Item No. 31760  
110 ct. refill pouch (12" x 12") Item No. 31761



- **Clorox Healthcare® Spore10 Defense™ Cleaner Disinfectant**  
EPA Reg. No 67619-40  
128 fl. oz. bottle, Item No. 32122

### **Notes (continued)**

*This list is missing disinfectant brands which had been previously recommended. Under the most recent protocols for durability testing, many cleaners failed, and therefore shortened the life of the material. We continually work to determine if other cleaners may be suitable, and to give our customers guidance for cleaners that support a long product life.*

*All disinfectants and cleaning agents contain chemicals that degrade your table upholstery to some extent. To promote a long product life we recommend you adhere to Midmark's [Upholstery Care and Maintenance Guidelines](#).*

<sup>1</sup> See current CDC Guidelines for Disinfection and Sterilization in Healthcare Facilities.