

Notes

Midmark's standard policy on disinfecting medical equipment, which is printed in the Care and Maintenance portion of the User Guides, is to use a solution of bleach and water mixed in a 1:10 ratio (10%) to disinfect the upholstery.¹ This should be followed with a clear water rinse and thorough drying of material. Any deviation from this recommendation could risk damage to the upholstery, possibly causing premature cracking, staining or discoloration.

For convenience you may wish to purchase a premixed disinfecting solution in spray or wipe form. Bleach based products manufactured with a 1 in 10 ratio (10%) have been shown to be compatible. The following off the shelf disinfecting solutions have been tested by an independent lab and deemed safe for use on our upholstery and all surfaces of our exam and procedure tables. This list supersedes all previous versions (before April 2018).

Here are some available premixes:

- **Clorox Healthcare™ Bleach Germicidal Wipes**
EPA Reg. No 67619-12
6" x 5" Clinical Wipes, Item No. 30577
6.75" x 9" Multipurpose Wipes, Item No. 35309
12" x 12" Terminal Wipes, Item No. 30358
12" x 12" Terminal Wipes Refill, Item No. 30359
- **Medtro® SUPER-CHLOR**
EPA Reg. No 69687-1
9" x 9" Wipes, Item No. SC110
12" x 12" Wipes, Item No. S112
- **Clorox Healthcare® Hydrogen Peroxide Cleaner Disinfectant**
EPA Reg. No 67619-25
6.75" x 9" Wipes, Item No. 30824
6.75" x 5.75" Wipes, Item No. 30825
12" x 11" Wipes, Item No. 30826
12" x 11" Wipes Refill, Item No. 30827
EPA Reg. No 67619-24
32 ounce Spray, Item No. 30828
32 ounce Spray Refill, Item No. 30829
- **PDI Sani-Cloth® Plus Germicidal Disposable Cloth**
EPA Reg. No 9480-6
6" x 6.75" Wipes, Large Canister (160 count) Item No. Q89072
7.5" x 15" Wipes, X-Large Canister (65 count) Item No. Q85084
- **Metrex™ CaviWipes®**
EPA Reg. No 46781-8
6" x 6.75" CaviWipes, (160 count) Item No. 13-1100
9" x 12" CaviWipesXL, (65 count) Item No. 13-1150
- **Alcavis® Bleach-Wipes**
EPA Reg. No 65787-2
8" x 10" Alcavis Bleach-Wipes, (100 count) Item No. 09041-100

Notes (continued)

This list is missing disinfectant brands which had been previously recommended. Under the most recent protocols for durability testing, many cleaners failed, and therefore shortened the life of the material. We continually work to determine if other cleaners may be suitable, and to give our customers guidance for cleaners that support a long product life.

All disinfectants and cleaning agents contain chemicals that degrade your table upholstery to some extent. To promote a long product life we recommend you adhere to Midmark's [Upholstery Care and Maintenance Guidelines](#).

¹ See current CDC Guidelines for Disinfection and Sterilization in Healthcare Facilities.