

Cylinder Mount Manifold

[93305369, 94305370, 94305371, 94305372, 94305373, 94305383, 94305384, 94305385, 94305389]



Caution

Remove all stand components and loose equipment before beginning installation of cylinder mount manifold.

To install (Read before installing)...

1. Place a mark on the post 28 inches up from floor level.
2. Carefully tip stand onto its side. **CAUTION:** stands with shelves may weigh as much as 50 pounds, two or more persons may be needed to tip and lower the stand.
3. Holding onto the post, near the 5-leg base, lift the base off the floor. While tightly grasping the post slightly twist the base until it becomes loose. Remove the base.
4. Slide the cylinder mount manifold over the post. Be sure top of cylinder mount manifold (label side) is facing away from base end. (Note: the cone point bolt may need to be backed out slightly to allow the cylinder mount manifold to slide freely over the post).
5. Position the cylinder mount manifold above the mark you placed on the post in step 1. Align the cylinder mount manifold's the gauge(s) face in the desired direction.
6. Using a ¼ inch hex wrench, fully tighten the cone point bolt.
7. Install the 5-leg base.
8. Turn stand upright.
9. Install cylinder(s) onto cylinder mount manifold.

Note: The five-leg base may have to be turned to allow the cylinder(s) to clear the leg(s).

