




# IQvitals® Power Supply Instructions for converting power cord



1. Locate the IQvitals® Power Supply (Figure 1)
2. Remove the plug-in prongs from the IQvitals® Power Supply by sliding the prongs upward. (Figure 1)

## Power Cord Kit

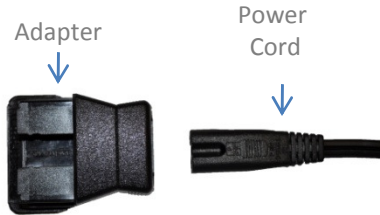

3. Remove the IQvitals® Power Cord Kit from the package.
4. Insert the power cord into the adapter. (Figure 2)
5. Slide the adapter on the IQvitals® Power Supply. (Figure 3)  
*Note: be sure to properly align the connectors.*

Figure 2

6. Retain the plug-in prongs in case you want to convert the power supply back to the wall plug power supply.



Figure 3

