



# IQvitals® Printer Mounting Instructions

## Instructions for Attaching to IQvitals® Mobile Cart

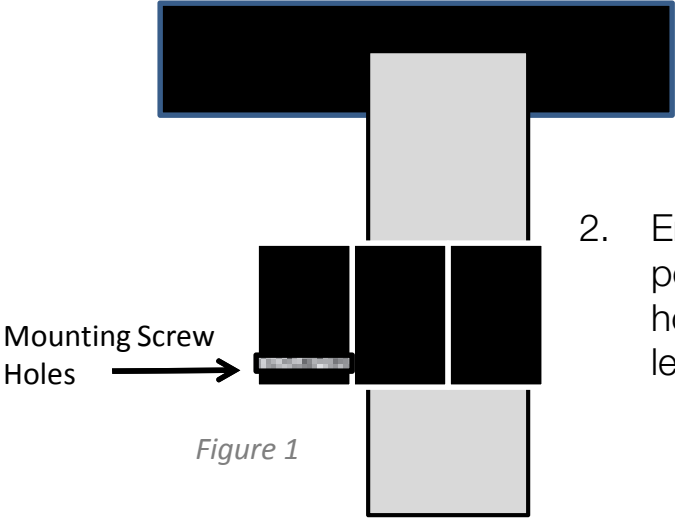


Figure 1

1. Have the Printer Mounting Bracket already attached to the cart. (Figure 1)
2. Ensure the printer mounting bracket is positioned with the mounting screw holes at the bottom and facing to the left side of the cart.

3. Open the printer door and remove paper if in printer.
4. Locate mounting screws and insert them into the holes at the back of printer. A Philips screwdriver is required. (Figure 2)

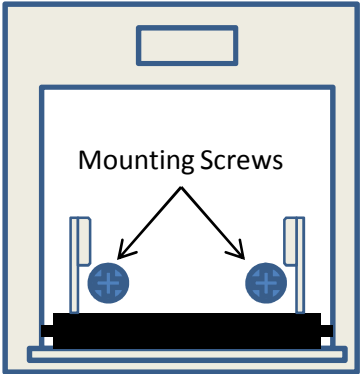


Figure 2

5. Position the printer such that the mounting screws align with the holes on the mounting bracket. (Figure 3)
6. Screw the printer mounting screws into the mounting bracket.

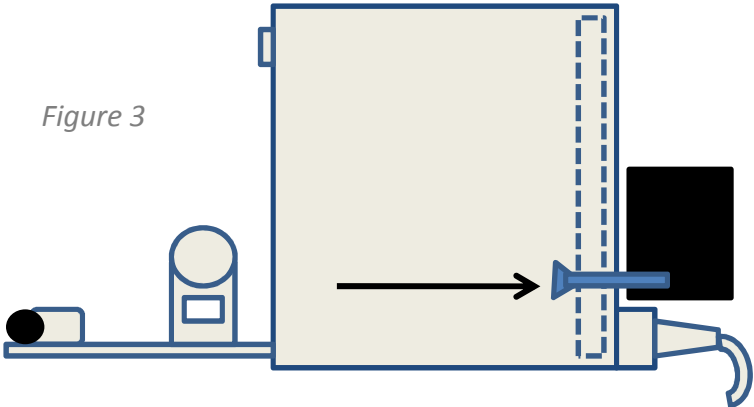


Figure 3

7. Insert paper and plug printer cable into IQvitals®.

