


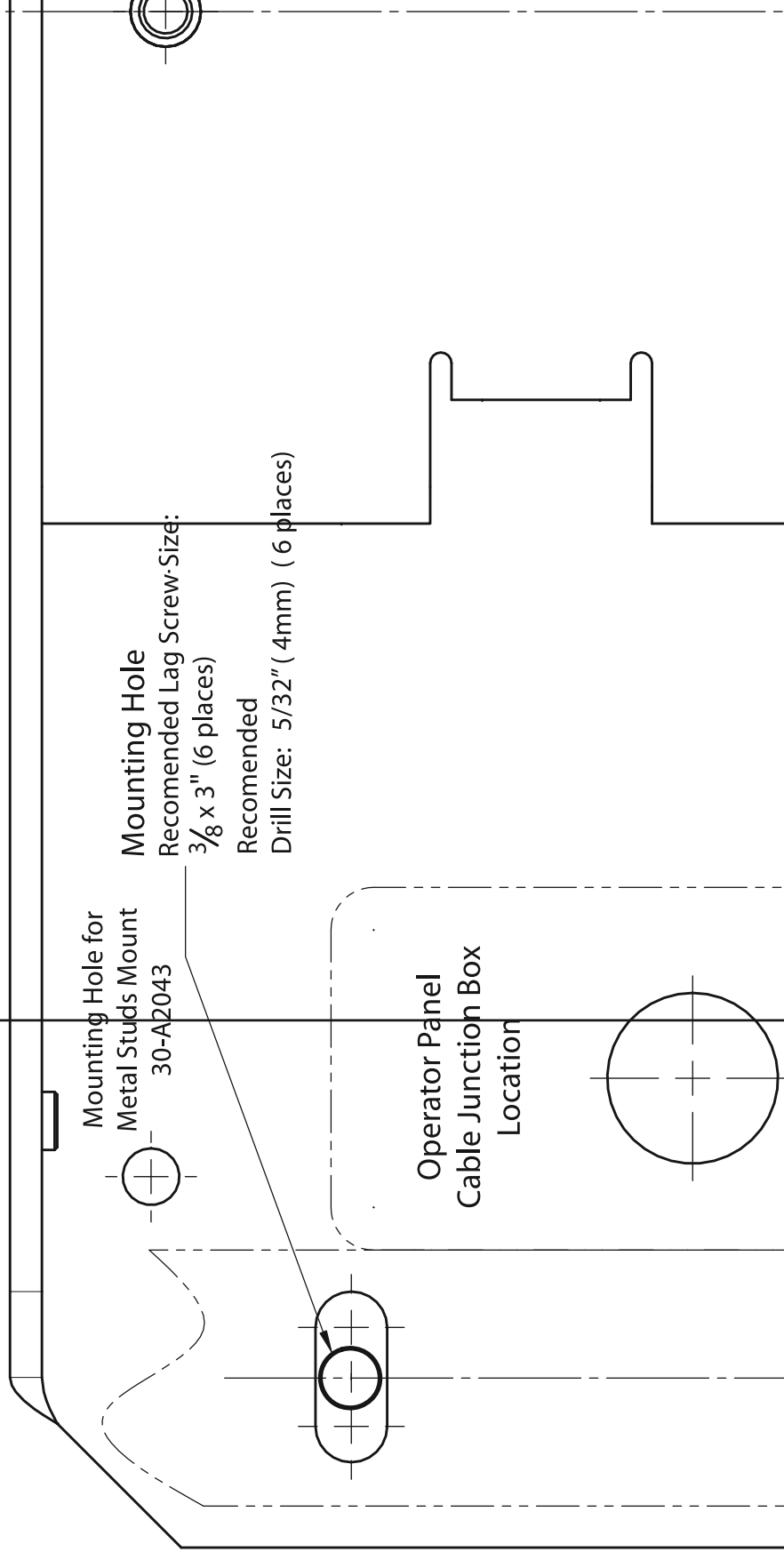


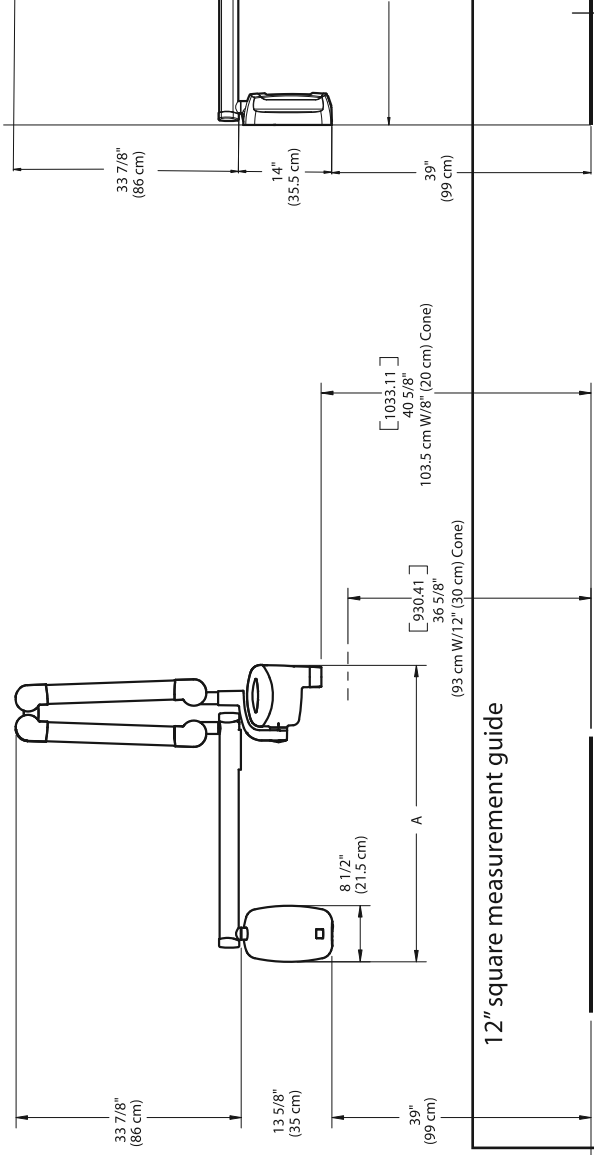


JB-70

PREVA

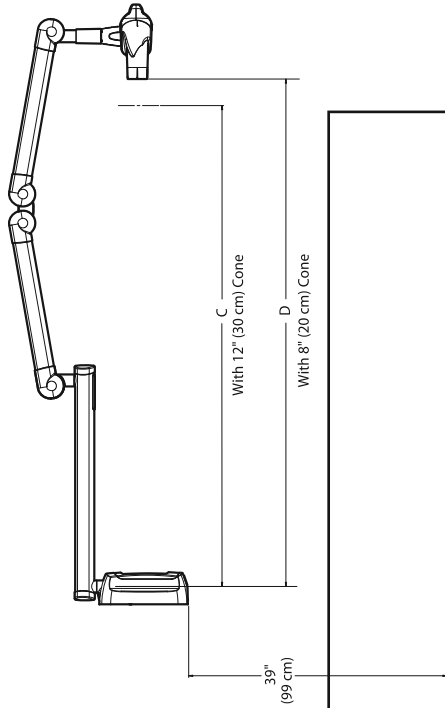

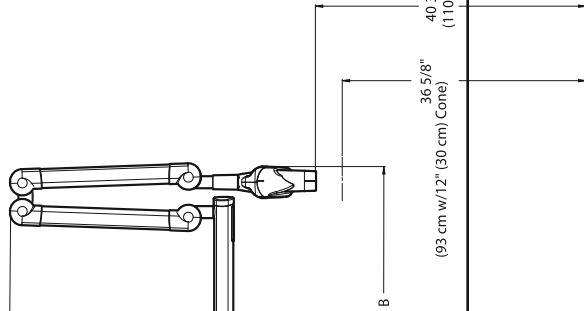

Catalog Number (Beginning with)	A	B	C	D
DCD5	24 5/8" (63 cm)	20 1/2" (52 cm)	52 1/2" (133 cm)	56 1/2" (144 cm)
DCV5 I7015 P7015				
DCD6	34 5/8" (88 cm)	30 1/2" (78 cm)	62 1/2" (159 cm)	66 1/2" (167 cm)
DCV6 I7016 P7016				
DCD7	44 5/8" (114 cm)	40 1/2" (103 cm)	72 1/2" (184 cm)	76 1/2" (194 cm)
DCV7 I7017 P7017				
DCD8	48 5/8" (124 cm)	44 1/2" (113 cm)	76 1/2" (194 cm)	80 1/2" (204 cm)
DCV8 P7018				

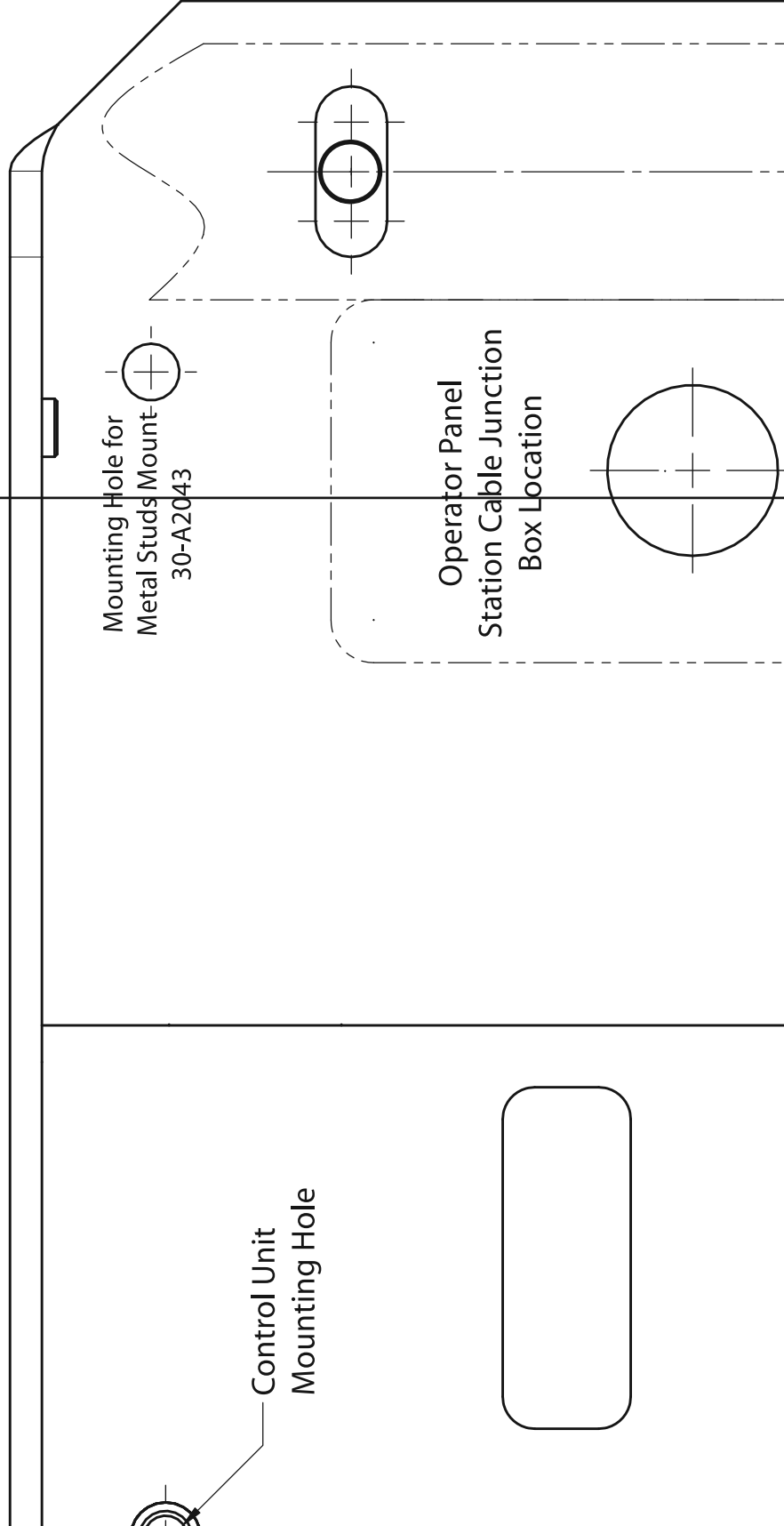

Mounting Hole for
Metal Studs Mount-
30-A2043

Operator Panel
Station Cable Junction
Box Location

Control Unit
Mounting Hole

Control Unit - Two

Electrical Power Junction Box Location
Junction Box Size: 4" x 4"

12" square measurement guide

Power Cable
Opening

Recommended Location
41" from Floor

Mounting Hole for
Metal Studs Mount
30-A2043


Recommended
40" f



Made in USA

Midmark Corporation • 1001 Asbury Drive Buffalo Grove IL 60089 • USA • 1-800-MIDMARK • midmark.com

o Stud Mount

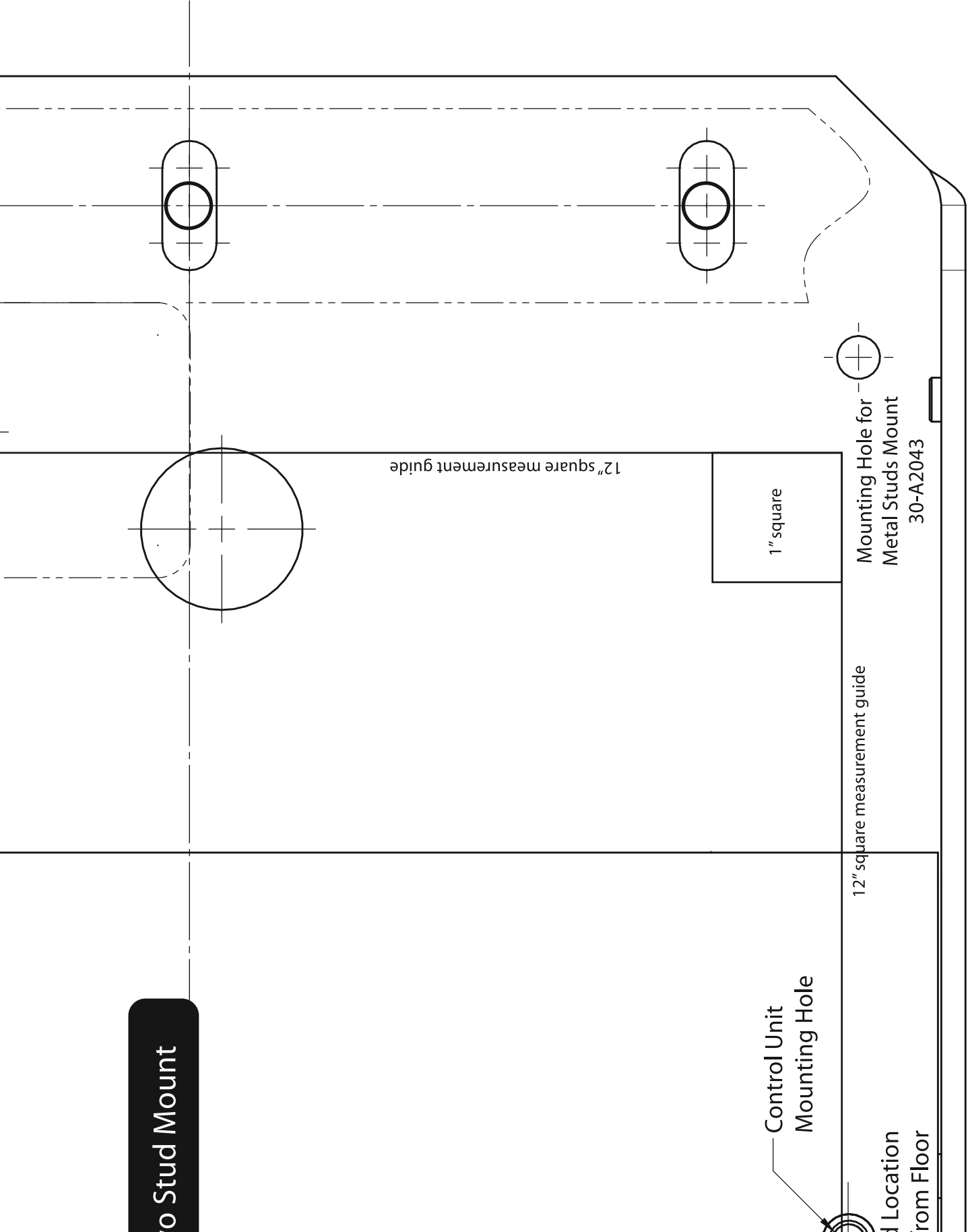

12" square measurement guide

1" square

12" square measurement guide

Mounting Hole for
Metal Studs Mount
30-A2043

Control Unit
Mounting Hole

Location
from Floor