

Tools required:  
#2 Phillips Drive Screw driver or #2 Square bit drive

Parts Included:  
(1) Multi-Purpose Shelf  
(2) Sets of Mounting Bracket Assemblies  
(4) 10-24 x 7/8" Long Phillips Drive Pan Head Machine Screws  
(4) 10-24 x 1" Long Phillips Drive Pan Head Machine Screws  
(1) Safety Strap

**1** Measuring up 2" from the top of the hub, attach screws (1 each) into the back of accessory mounting bracket.

**2** Attach CPU accessory securely to mounting brackets through screw holes with (4) screws.

**3** Undo security strap

**4** Place CPU into MPS-1 holder. Secure Security Straps (see Fig. A).

**5** If using a printer, reposition the security strap from the horizontal position to the vertical position (see Fig. B).

