



Tools required:
Phillips head screwdriver or #2 Square drive bit

Parts Included:
(2) Accessory Mounting Brackets
(2) #10-24 x 7/8" Long Phillips Drive Pan Head Machine Screws
(2) #10-24 x 1" Long Phillips Drive Pan Head Machine Screws
(1) Container Holder
(1) Sharps Container - Mailbox Lid

- 1 Run screws into accessory mounting bracket to within 1/16" of the bracket surface
- 2 Securely attach accessory mounting bracket to the table column at the desired height
- 3 Attach container hanger to accessory mounting bracket with (2) screws provided).

Slip the SC (Sharps Container) accessory onto the wire bracket.

