

Tools required:  
1/8" Allen Drive

Parts Included:  
(1) Shelf Assembly

- 1** Remove (4) four #10-24 x 5/8" Long Flat Head Socket Cap Screws from the side panels of Upper Track Assembly (See Fig. A)
- 2** Place Shelf Assembly on upper track assembly (See Fig. B).
- 3** Make sure that the four (4) holes on the Shelf Assembly are aligned with the four holes on the Upper Track Assembly (See Fig C).

Fasten Shelf Assembly to Upper Track Assembly with (4) four #10-24 x 5/8" Long Flat Head Socket Cap Screws (See Fig. C).

