

OPERATING INSTRUCTIONS

for P/N 152134

DOUBLE ARTICULATING HEADREST

The KNIGHT Double Articulating Headrest is offered to our customers as an alternative to conventional headrest designs. This headrest is adjustable to practically any position that might be encountered by an operator.

1. HEIGHT ADJUSTMENT

The headrest height can be easily adjusted by simply pushing in or pulling out on the headrest assembly, causing the tang to slide in and out of the chair back (See Figure 1).

2. DOUBLE ARTICULATING ADJUSTMENT

To articulate the headrest, release the locking lever while supporting the headrest (See Figure 1). Position the headrest support in the desired location and lock in place by closing the locking lever. The cushion articulates freely to support the head in a comfortable position.

3. LOCKING FORCE ADJUSTMENT

The locking force is preset and should not require adjustment. However, if adjustment is desired please follow these steps:

- 1) Unlock the headrest support by releasing the locking lever (See Figure 1).
- 2) Locate the adjustment nut on the underside of the headrest support (See Figure 2). Using the adjustment tool #121429 provided and a pair of pliers, turn the nut slightly (less than 1/8 turn). *NOTE: Clockwise will increase locking force and counter clockwise will decrease locking force.*
- 3) Lock the headrest support by closing the locking lever and check the headrest to make sure it will provide proper support.
- 4) Repeat these steps until the desired locking force has been achieved.

For additional information, refer to the chair operation and care manual or contact KNIGHT Mfg.

Knight[®]
by Midmark

Midmark Corporation • P O Box 286 • 60 Vista Drive
Versailles • Ohio • 45380-0286 • www.midmark.com
Phone: 937-526-3662 Fax: 937-526-5542

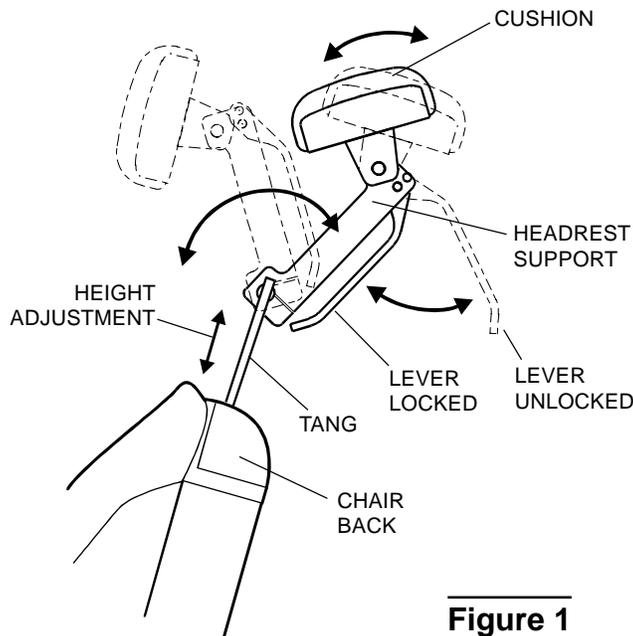


Figure 1

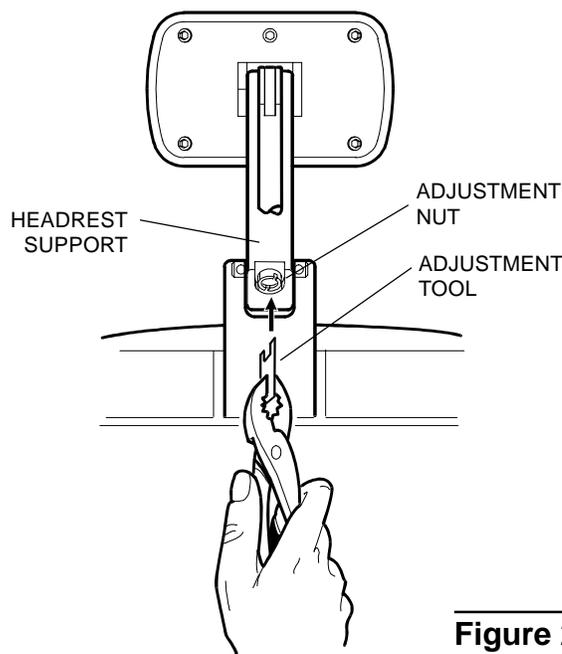


Figure 2