


VistaCool™

Adjustable Tank Stands

The VistaCool system should be set on a hard, level surface and installed so that the top of the water tank is above the coolant overflow fitting on the drain adapter assembly, and below the autoclave outlet. If the tank is below the level of the coolant overflow fitting, the system may not operate properly. Installing a tank at the wrong height may create resistance for proper water flow, since the system is designed to take advantage of gravity flow.

Adjustable tank stands for installing the VistaCool at the correct height are available as accessories when needed. These stands raise the tank height between ½-5" in half-inch increments.

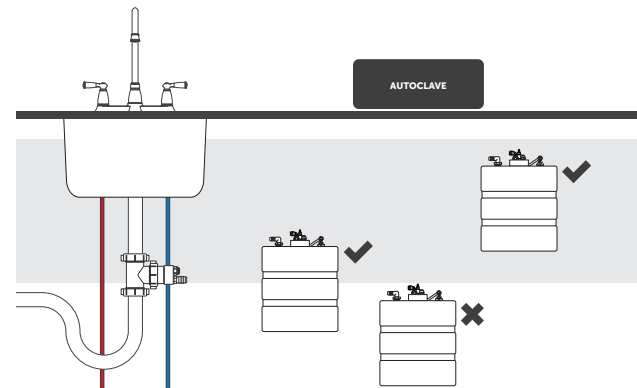

S7545 - Single Tank Stand with adjustable straps



S7547 - Double Tank Stand with adjustable straps

Instructions

- 1 Measure to determine how much additional height is required for the top of the VistaCool water tank to be above the coolant overflow fitting on the drain adapter assembly. Be sure the top of the tank is below the autoclave outlet.
- 2 Select the appropriate orientation for the adjustable tank stand. Note that because of how the openings in the stands are positioned, one orientation raises the water tank in even, one-inch increments (1", 2", etc.). Use the inverted orientation if you need to raise the tank in a ½" increment (½", 1-½", 2-½", etc.)
- 3 Insert the included metal "straps" into the appropriate openings to achieve the correct height. Be sure that the tank sets level and securely in the tank stand.

Vista Research Group

VistaCool.com

P.O. Box 321
Ashland, Ohio 44805-0321
United States of America

PHONE (419) 281-3927
FAX (419) 281-7380

DISTRIBUTED BY

Midmark Corp.
1-800-MIDMARK
midmark.com

SciCan Ltd.
1-800-572-1211
scican.com